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SAKAMOTO MODEL
CORPORATION

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SAKAMOTO MODEL
CORPORATION

User's Manual (191223)

●Introduction/ Table of Contents	1
●Purpose	2
●Content	3
●Precautions concerning handling	4
●Usage	5-7
●Storage method	7-9

* For the usage of the attached I.V. Pad, refer to the user's manual of I.V. Pad, which is a separate manual packed together with this model.

! Note	Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor.
-------------------------	--

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Scan this QRcode with your device to see our homepage.

Introduction

Thank you for purchasing our product, “New KOHARU.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

Introduction／Table of Contents.....	1
Purpose.....	2
Content.....	3
Precautions concerning handling.....	4
Usage.....	5-7
Storage method.....	7-9

* For the usage of the attached I.V. Pad, refer to the user’s manual of I.V. Pad, which is a separate manual packed together with this model.

Purpose

Training can be performed for the following items:

- Posture change
- Intravenous injection / blood sampling
- Gastrostomy care
- Artificial anus care
- Nightwear change
- Whole-body wiping
- Urethral catheterization
- Enema
- Intestine/gastric lavage
- Restriction
- Bandaging
- Intramuscular injection
- Oral hygiene
- Total parenteral nutrition
- Tracheotomy care
- Carrying
- Suctioning
- Nutrition (oral/nasal)
- Disimpaction
- Assisting excretion
- Fomentation
- Shampooing

Table of contents

1. Main unit		7. Exclusive Tape	2 pieces
2. I.V. Pad	1 set	8. Simulated gastric fistula catheter,	
3. Wig for shampooing	1 piece	Drainage bag	1 set
4. Pajamas	1 suit	9. Pseudo tracheal cannula	1 piece
5. Silicon oil	1 bottle	10. Catheter	1 set
6. Baby powder	1 bottle		

(14 Fr balloon catheter, 10 mm gastric catheter, nutrition catheter No.4, 15EG Nelaton catheter, 8EG Nelaton catheter)

Replacement parts (Sold Separately)

M102-1 Male Genitalia Model for “New TOMOKO”

Accessories: an irrigator and intestine

Precautions Concerning handling

1. Do not put the model on print including newspapers because it is made of silicon and will easily become dirty or stained. Clean dirt from the skin by wiping it gently using neutral detergents or alcohol-based chemicals.
2. Do not use any permanent markers or inks on the model because they will be absorbed into the skin and become impossible to remove.
3. When inserting the catheter, apply the attached silicon oil to the catheter to increase smoothness.
4. Moving joints unreasonably might lead to damage.
(If anything abnormal is detected during use of the model, notify us of it immediately.)

5. If somewhere on the model is sticky, apply the accessory powder lightly.
Using the model without taking the necessary action might lead to breakage
6. Do not use the attached silicon oil on the epidermis of the injection model such as the attached I.V. Pad.
7. Do not use pseudo tracheal cannula for humans.
8. After finish shampoo, please remove and dry the wig.
9. If anything strange is detected when this model is being used, stop using it and notify us as soon as possible.
10. Do not hand head and feet when you carry this model.
11. This product is a simulator for users to perform advanced training. Nonetheless, it does not guarantee that a similar effect can be obtained by actual treatment.
12. Note in advance that this product is subject to change without notice to improve it.

Usage

This section describes procedures for the major practice items.
Read them to perform training.

1. Muscle injection

Intramuscle injection (subcutaneous injection) can be performed in the four (total eight) regions of the shoulders, upper arms, thighs, and buttocks.

The pad is made of silicon, including a sponge in it.

Bring the sponge out of the pad, squeeze it, and dry it in the shade. Return it to the epidermis, add the powder, and replace it in position. (This is illustrated in the section on the storage method.)

2. Intravenous injection / blood sampling

When performing training, set the accessory, I.V. Pad, on the model.

For the usage of I.V. Pad, refer to the separate user's manual of I.V. Pad packaged together with this model.


3. Gastrostomy care

The abdominal region is equipped with a stoma for gastrostomy. Apply the lubricant oil before inserting simulated gastric fistula catheter.

Note that injecting fluids is not allowed.

Please use tap or pure water when you inject.

※Preparation of Gastrostomy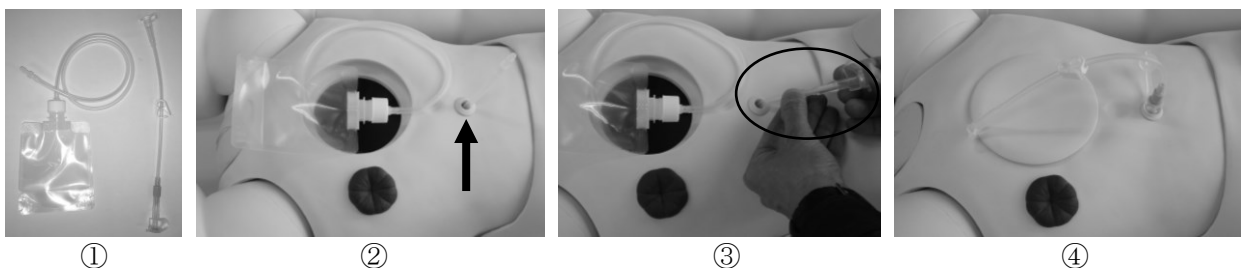

- ①Please arrange drainage bag and gastric catheter.
- ②Pull up head of tube from gastric fistula.
- ③Connect gastric catheter.
- ④Put drainage bag in abdomen then close the lid.

4. Total parenteral nutrition

The model is equipped with a hole around the infraclavicular vein. Apply the lubricant oil before inserting the tube. Note that injecting fluids is not allowed.

5. Artificial anus care

The stoma part is bonded to the model. Do not attempt to remove it from the main unit.

6. Tracheotomy care

Remove the silicon plug from the tracheotomy site, and set the pseudo tracheal cannula for training. After the training ends, replace the silicon plug and store it.

7. Catheterization (suction / urethral catheterization)

For each procedure, use an applicable catheter.

Apply the lubricant oil, and insert the catheter delicately.

After the training ends, remove the lids on the chest and stomach region to dispose of the solution remaining in the stomach, intestine, and bladder directly.

Remove the bladder together with the external genitalia, and tilt it to drain the water.

Suction training using simulated sputum is not allowed.

(To perform such training, use M175 Sakamoto Suction Simulator.)

8. Urethral catheterization

Take off the lid on the abdominal region to remove the tube for urethral catheterization. Make sure that this tube is inserted into the tip of the irrigator's tube securely.

Hang the irrigator on the stand, pour water into the bag, and open the clamp.

Apply the lubricant oil to the catheter tip, and insert it to train in urethral catheterization.

9. Enema

Apply the lubricant oil to the catheter, and insert it at a low speed.

To prevent injected fluid from flowing backward, the model is equipped with a valve in front of the rectum.

Although you will feel slight resistance when inserting the catheter, fluids can be injected once it passes through the valve.

Do not perform enema using glycerin.

10. Manual disimpaction



Manual disimpaction training using this model is designed to learn the procedure. Do not perform the training by inserting something like simulated stool because it might lead to the breakage of the epidermis. (It is possible to conduct training by attaching the separately sold M169 Fit-on Manual Disimpaction Simulator.)

11. Shampooing

Put the wig on and fix it with the accessory cord to train in shampooing. Since the wig has artificial hair made of nylon, do not use a hair drier to dry it with hot air

Storage method

1. After training in intramuscle injection, bring the sponge out of the pad, squeeze the injection solution (water only), and dry it in the shade. Then, add the powder, and replace the pad in position.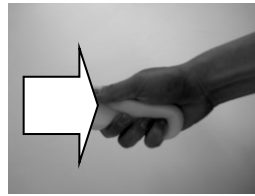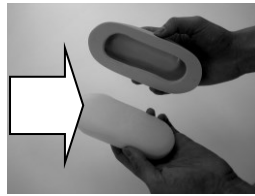

2. Do not hand head and feet when you carry this model. Neck and waist might be damaged.
3. After using the stomach or intestine, be sure to empty it out, and dry it to the original state before storing the model.
 - ①When you flex and more model's wrist, please do not move over human range of Motion.
 - ②Neck can bend in all direction. Please do not bend over range of motion.
 - ③Finger joint is exquisitely. Please handle with care.
4. When the model gets dirty, wipe off the dirt using a soft cloth with a neutral detergent. After that, wipe the model with a cloth moistened with water.
5. Note that some silicon chemical might ooze and stain the nightwear if the model is stored without removing the nightwear.
6. Lay the model with each joint stretched before storing it.
If there is more than one model, do not pile them up but store them one by one.
7. If the epidermis is sticky, be sure to apply the attached powder before storing the model.
8. Do not use silicon oil on I.V. Pad.
If silicon oil gets on the model by mistake, wipe it off with a dry cloth before storing it.
Otherwise, the quality of the epidermis might be degraded.
9. Do not store the model anywhere that is exposed to direct sunlight or is hot and humid.

STORING METHOD


~For long and safe using~

Do not store like these posture for long time:

Do not store for long time with the neck is bent.


Do not store for long time with the arms are bent.

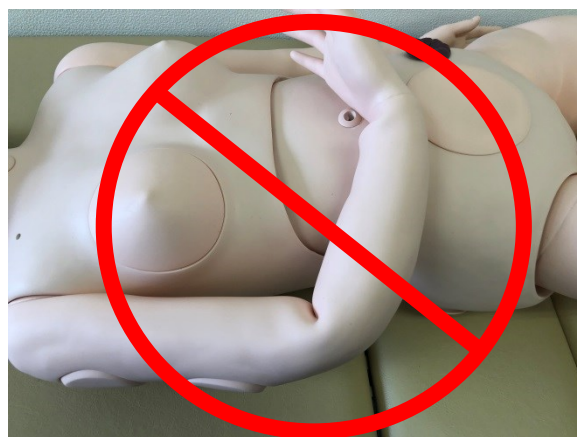Do not store for long time with the knees are bent.


Do not store for long time keep sitting.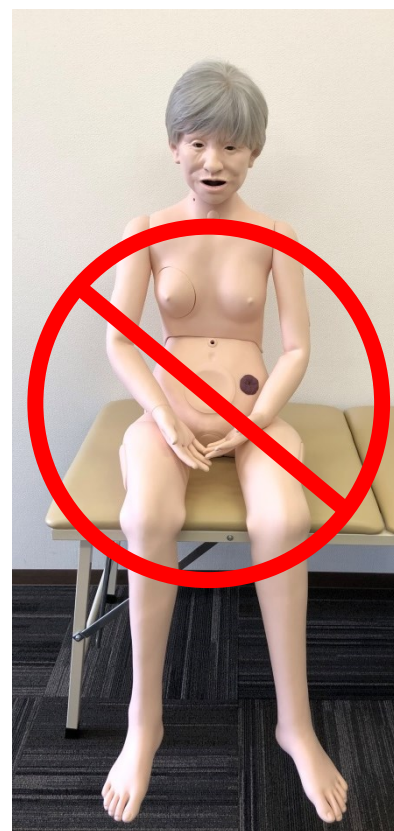

M104

Keep the whole body in flat like photo when storing

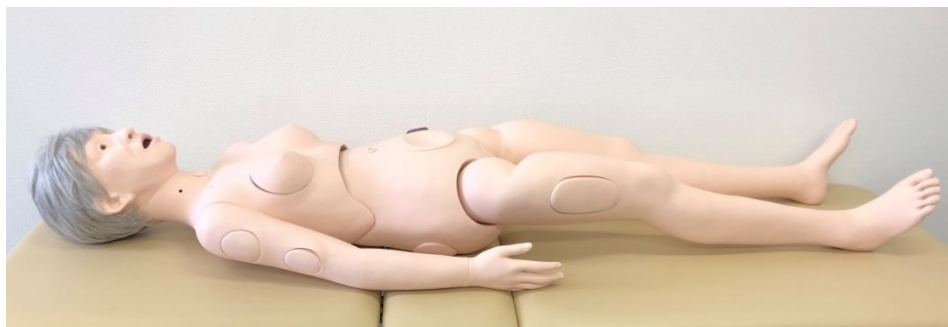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URL <http://sakamoto-model.co.jp>

E-Mail trade@sakamoto-model.com

Product specifications and appearance are subject to
change for improvement without notice.

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- Some of the components used in this product are manufactured by other companies, and product names, etc. for these components may be trademarks or registered trademarks of the manufacturers.