M201

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User's Manual (260602)

— I N D E X —

● Introduction / Table of Contents	1
● Purpose / Contents	2
● Cautions on use / Storage Instructions	3
● Instructions for Use	4

!
Note

Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor.

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Scan this QRcode with your device to see our homepage.

Introduction

Thank you for purchasing our product, “Pregnancy Experience Suit”.
Read this user’s manuals carefully in order to use the product for a long
time appropriately.

This product is closely checked before factory shipment. However, when
this product arrives, open it soon, and check if anything on the main unit
and accessories are missing or broken.

Table of Contents

Introduction／Table of Contents.....	1
Purpose／Content.....	2
Cautions on use／Storage Instructions.....	3
Instructions for use.....	4

Purpose

This suit is used simulate pregnancy

Contents

- | | | |
|---|--|-------|
| 1 | Pregnancy simulation unit | Qty.1 |
| 2 | Chest weight bag
(Includes shape-holding sponge and 750g weight x2) | Qty.2 |
| | Stomach weight bag
(Includes shape-holding sponge and 750g weight x8) | Qty.1 |
| 3 | Storage bag | Qty.1 |

1

2

3

Cautions on use

- 1 When training and simulating the experience of pregnancy, always carry out the training with at least two assistants or supporting staff.
- 2 Be careful not to fall down when walking on roads or uneven surfaces.
In addition, the person simulating the experience of pregnancy may also accidentally drop objects and therefore should exercise caution.
- 3 Approx. 9kg when all weights are fitted
When fitting, adjust to a weight which is possible to bear.
- 4 The simulation suit is washable by hand. Be sure to look at the tag inside the suit and follow the instructions.
- 5 Note that the fasteners, etc., may become brittle after washing the suit multiple times.
- 6 If a defect or problem with the product is discovered during use, stop using the product immediately and contact Sakamoto Model or an authorized dealer.
- 7 This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee the same results during actual treatment.
- 8 Please note that the specifications of this product may be changed without prior notice in order to make improvements to the product.


Storage Instructions

- 1 The simulation suit is washable by hand. Be sure to look at the tag inside the suit and follow the instructions.
- 2 Fold up the pregnancy suit and place it into the storage bag
- 3 Avoid storing the product in direct sunlight or in locations with high temperature and humidity.

Instructions for Use

Always carry out the training with at least two assistants or supporting staff.

- 1 Place the shoulder belt onto your shoulders.
- 2 Attach the waist belt and adjust it to fit your body shape.
- 3 Attach the chest belt and adjust it to fit your body shape.

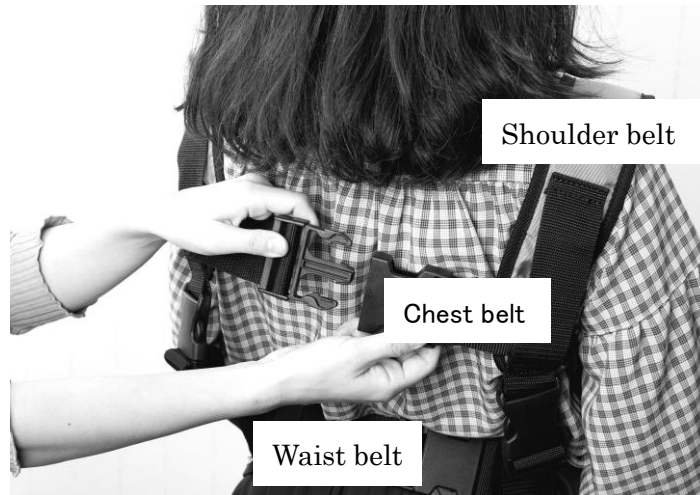- 4 Adjust the weights around the chest and waist as desired

Each weight weighs approximately 750g.

(There are 8 waist weights and 2 chest weights.)

When fully fitted

When apron (sold separately) is worn

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka
562-0035,JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com