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# Ultrasound Guide Arterial Puncture Trainer

## User's Manual

Only use **water**<sup>\*1</sup> or **artificial blood**<sup>\*2</sup>  
for carrying out puncture training using this product!  
Furthermore, do not use syringes with bent needle tips!

\*1 Use either tap water or distilled water for injection use.

\*2 To create artificial blood, add 1 to 2 drops of the artificial blood concentrate to water.

The accessory AC adapter is of 12V specifications. Do not substitute it with AC adaptors manufactured by other companies, or use AC adaptors manufactured by Sakamoto Model together with other companies' AC adaptors. Doing so may result in incorrect operation or damage to the product.

### Table of contents

1	Introduction	24	6	How to use	30
2	Product contents	25	7	Cleaning and storage	41
3	Cautions on use	26	8	Troubleshooting	42
4	Product applications	28	9	Specifications	43
5	Part names	29			

# 1 Introduction

## Thank you very much for purchasing the "Ultrasound Guide Arterial Puncture Trainer".

This user's manual contains cautions and notes on how to use this product safely. Please read this manual carefully in order to ensure that you know how to use the product safely.

In addition, after reading the manual, keep it in a safe place where it can be referred to at any time later on.

After purchasing the product, check the package contents carefully to make sure that nothing is missing or broken. If you notice any problems, contact Sakamoto Model.

### ■ Guide to product's introductory page

---

You can view this product's introductory page on our company's website by using either the following URL or the QR code\*.

- Sakamoto Model Corporation's Introductory page for the "Ultrasound Guide Arterial Puncture Trainer"

<https://sakamoto-model.com/echo/m210/>


\* QR code is a registered trademark of Denso Wave Inc.


## 2 Product contents


If no quantity is listed, the quantity included is one.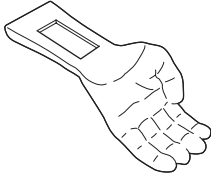


Wrist model

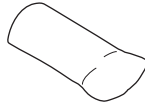
Puncture pad

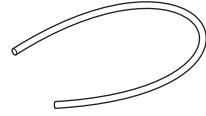
Blood vessel tube

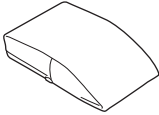
Special pillow

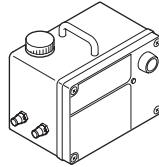Tank with integrated pump

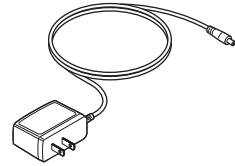
AC adapter\*

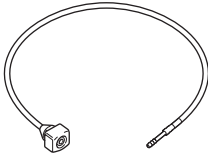
Blood vessel connection tubes  
(Pink mark or light-blue mark each ×1)

Cup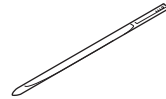

Blood vessel threader

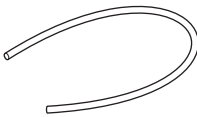Replacement blood vessel tube ×10

### CAUTION

\* The accessory AC adapter is of 12V specifications. Do not substitute it with AC adaptors manufactured by other companies, or use AC adaptors manufactured by Sakamoto Model together with other companies' AC adaptors. Doing so may result in incorrect operation or damage to the product.

### TIP

This product does not include ultrasound equipment.

## 3 Cautions on use

In order to use this product safely, be sure to observe the following.  
(Items which are particularly important are marked with " ■ ".)

### ● **Do not subject the product to impacts or other forces.**

- Do not drop.
- Do not bump against solid objects.
- Do not apply excessive loads.

If any of the above occur, damage may result.

### ● **Handle the puncture pad gently.**

- Do not touch it with sharp objects such as knives, scissors.
- Do not pull or twist it with a strong force.

### ● **Always use fresh needle tips in the syringes when practising.**

If you use a syringe with a needle that has a bent or damaged tip, it may cause the blood vessels to become damaged or the puncture pad to become degraded.

### ● **Only use water or artificial blood as the fluid for puncturing.**

If a fluid other than those mentioned above is used, damage or disagreeable odors may result.

Use either tap water or distilled water for injection use.

To create artificial blood, add 1 to 2 drops of the artificial blood concentrate to water.

### ● **Do not disassemble or modify the product.**

If you attempt to disassemble or modify the product, it may become damaged or problems with operation may result. Never disassemble or modify the product.

### ● **Do not write on the Puncture pad with items such as ballpoint pens.**

If you write on the Puncture pad using a ballpoint pen or similar object, the surface will become smeared with ink which cannot be removed.

### ● **Do not use continuously for long periods.**

As a means to protect the product, avoid using it for long periods of time. (No more than 3 hours at a time is recommended.)

- **The AC adapter is for use only with this product.**

The accessory AC adapter is of 12V specifications. Do not substitute it with AC adaptors manufactured by other companies, or use AC adaptors manufactured by Sakamoto Model together with other companies' AC adaptors. Doing so may result in incorrect operation or damage to the product.

- **Hold the body of the AC adapter when removing it from an outlet.**

When disconnecting the AC adapter from the wall outlet, hold it by the AC adapter body, not by the cable. If you hold by the cable when disconnecting it, the cable may become damaged, and it may lead to electric shocks or fire occurring. In addition, do not connect or disconnect the cable with wet hands.

- **If you notice any problems when using the product, stop using it.**

If problems occur with the product during use, stop using it, turn off the power, and disconnect the power cable from the electrical outlet. Then contact Sakamoto Model. If you continue using the product, accidents may result.

- **Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.**

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, problems with operation may result.

- **There are limits to the effectiveness of simulator training.**

This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee that training using this product will provide the same level of expertise as actual treatment.

- **Do not use this product for other applications.**

Do not use this product for applications other than the original application for which it is designed. The product may become damaged or injury may result.

## 4 Product applications

This product can be used to carry out the following training.

### ■ Training

---

The following training for radial arteries and ulnar arteries can be carried out.

- **Puncturing**
- **Drawing blood**
- **Maintenance and indwelling of lines**
- **Catheter insertion \***

### ■ Features

---



- The pump creates the effect of blood circulating through the veins, so that pulse tactility and backflow can be checked.
- Ultrasonographic scanning can be carried out on major axes and minor axes for ultrasound-guided puncturing.

#### CAUTION

- \* Be careful not to insert the catheter into the tank with integrated pump. If the catheter is inserted into the pump, damage may result.

# 5 Part names


## ■ Tank with integrated pump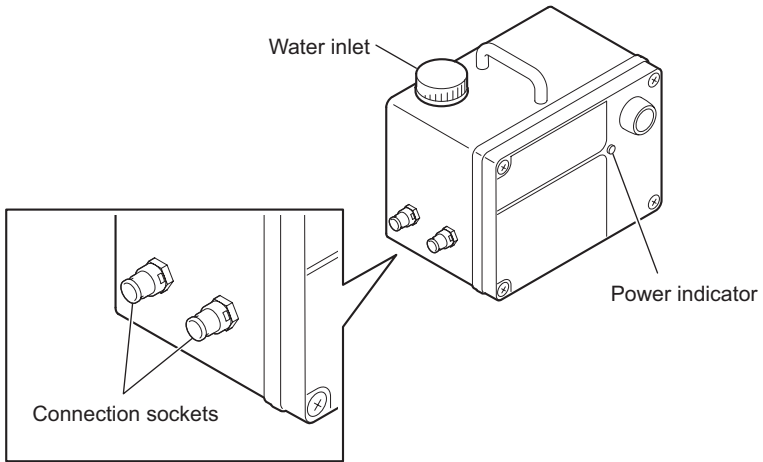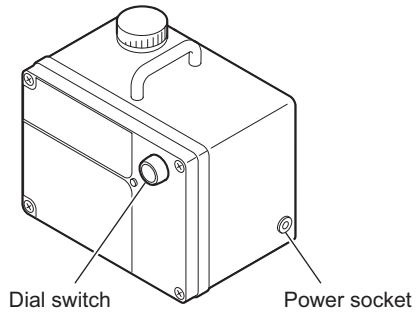

# 6 How to use

## ■ Preparation


- 1 Pass the end of the blood vessel tube through the blood vessel threader.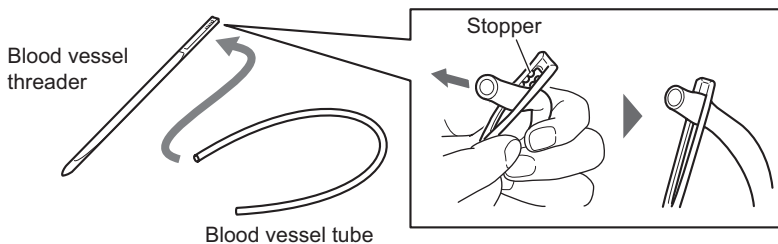

### ● TIP

Widen the hole in the blood vessel threader, and insert the end of the blood vessel tube so that it hooks onto the stopper.


- 2 Pass the blood vessel threader into the through hole and pull the blood vessel tube through the puncture pad.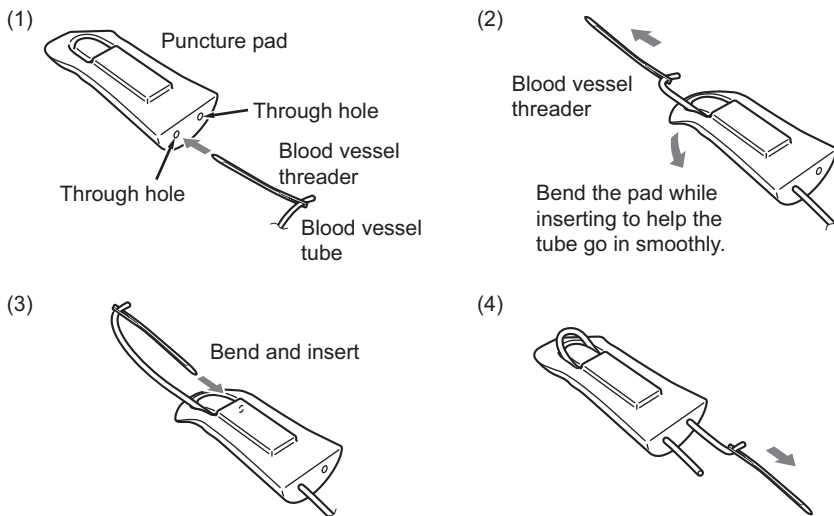


### ● TIP

The tube can be inserted into either through hole.

- 3** After aligning the protruding lengths of the tube with each other, fit the bent section into the groove in the puncture pad.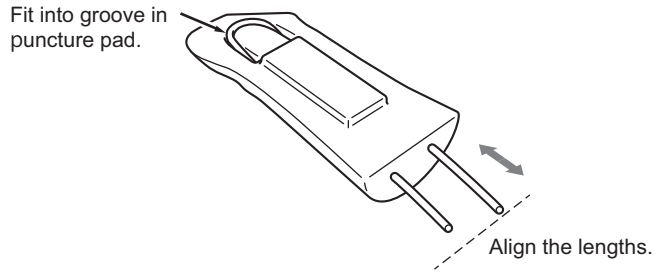


- 4** Insert the blood vessel connection tubes into the two ends of the blood vessel tube.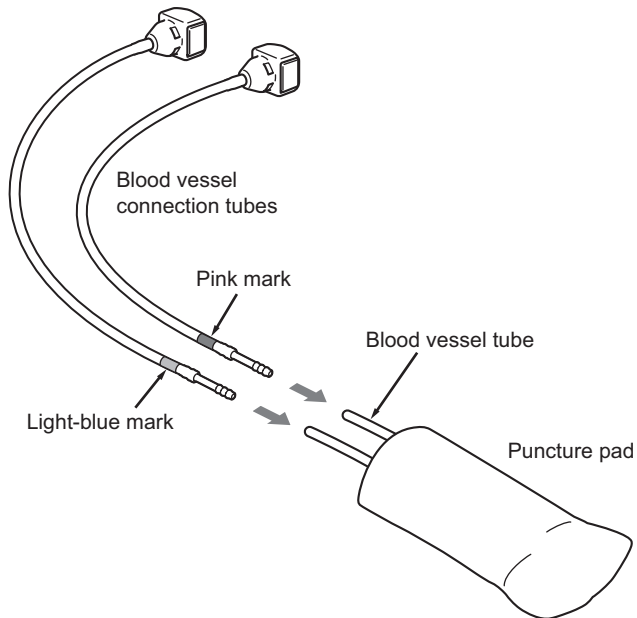

**TIP**

Insert the blood vessel connection tube with the pink mark into the radial artery and insert the blood vessel connection tube with the blue mark into the ulnar artery as shown in the diagram.

**5** Insert the blood vessel connection tubes into the connection sockets of the tank with integrated pump.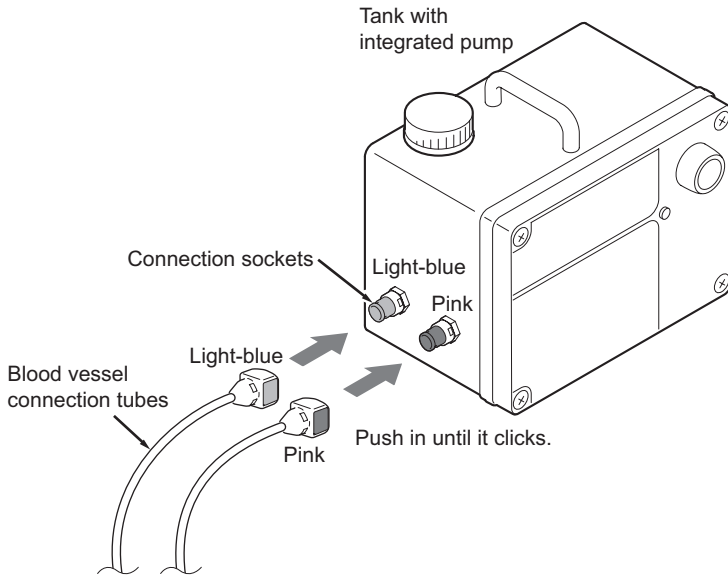

**TIP**


Insert the blood vessel connection tubes into the connection sockets so that the colors match as shown in the diagram.

**CAUTION**

- Push the blood vessel connection tubes firmly into the sockets until they click into place. If they are not properly connected, water may leak out from the connection sockets.
- When disconnecting the blood vessel connection tubes, press the buttons to unlock the connectors and then remove the tubes. If you try to pull out the blood vessel connection tubes by force, they will become damaged and it will cause water to leak out. (Refer to "Putting away" (P.39))


## 6 Install the puncture pad to the wrist model.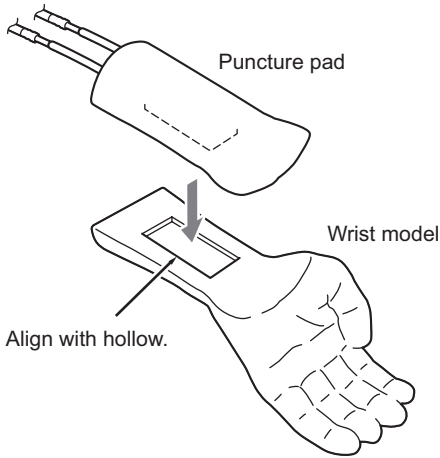

### CAUTION

- Be sure to place the puncture pad onto the wrist model when carrying out training. If training is carried out using only the puncture pad, accidental injuries such as being pierced by the injection needle or damage may result.
- Route the blood vessel tube and blood vessel connection tubes carefully so that they do not become bent.

## 7 Place on top of the special pillow.

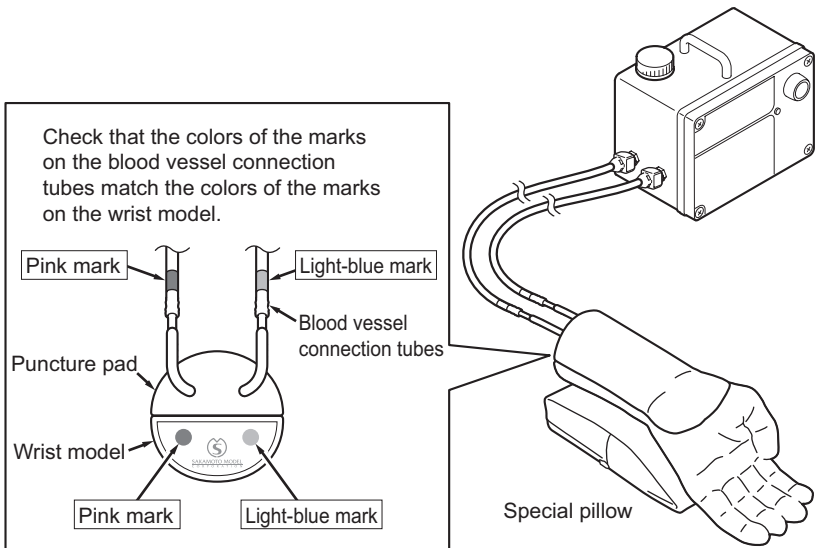
**8** Pour 200 ml of water into the cup.

**TIP**

- Use either tap water or distilled water for injection use. Artificial blood can also be used. To create artificial blood, add 1 to 2 drops of the artificial blood concentrate to water.
- In the following procedure, the term "water" is used exclusively.


**9** Pour the water into the water inlet of the tank with integrated pump, and then tighten the cap securely.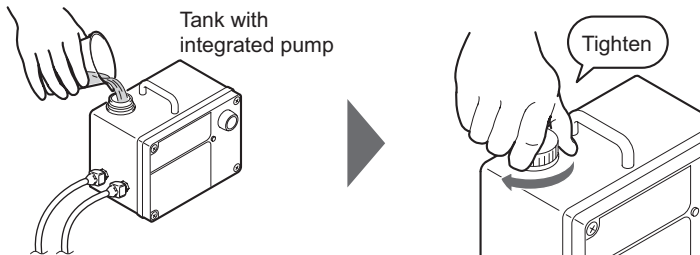

**CAUTION**

- Be careful not to spill the water. If the water gets spilled, wipe it away immediately. Otherwise electric shocks may occur.
- Tighten the cap securely. If the cap is not tightened securely, water may leak out.

- 10** Check that the dial switch is at the off position, and then connect the AC adapter to the tank with integrated pump and to an electrical outlet.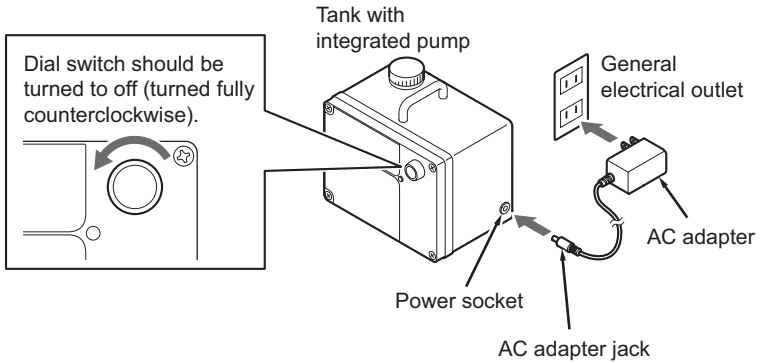


**CAUTION**

- The accessory AC adapter is of 12V specifications. Do not substitute it with AC adaptors manufactured by other companies, or use AC adaptors manufactured by Sakamoto Model together with other companies' AC adaptors. Doing so may result in incorrect operation or damage to the product.
- Do not handle the AC adapter with wet hands. Otherwise electric shocks may occur.


## ■ Training

Turn the dial switch of the tank with integrated pump to the right to turn on the power.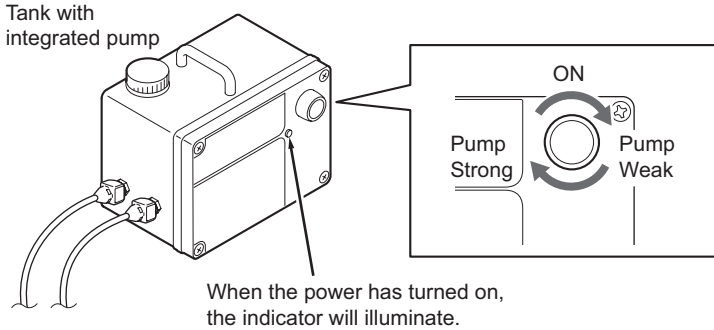

### ⚠ CAUTION

- Do not bend the blood vessel tube and the blood vessel connection tubes or place objects on top of them while the pump is operating. Extra loads will be placed on the pump, and problems may result.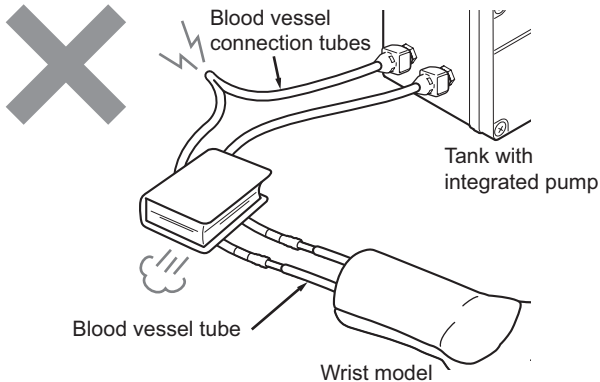

- When using a catheter:  
Be careful not to insert the catheter into the tank with integrated pump. If the catheter is inserted into the pump, damage may result.


### ● TIP

The water from the pump is not sent uniformly, but intermittently. This is not a sign of a problem.

## < For training in ultrasound-guided puncturing >

You can use the ultrasound equipment which you have on hand to carry out ultrasound-guided puncturing.


### 1 Apply ultrasonograph gel to the area to be monitored.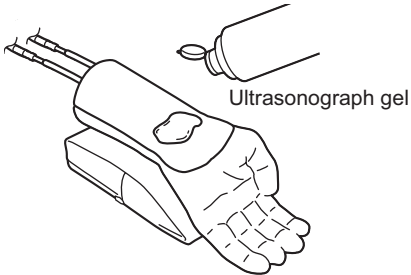

#### ⚠ CAUTION

- Use gel which is used for actual ultrasonographs.
- Do not leave the Puncture pad with gel still applied for so long that the gel either solidifies or evaporates. The Puncture pad may become stained.

### 2 Touch the probe against the puncture pad to carry out training in ultrasonographic scanning.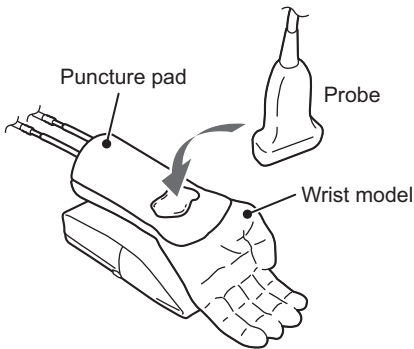

#### ● TIP


Adjust the gain setting for the ultrasound if necessary.

#### ⚠ CAUTION


- If artificial blood gets onto the probe, it may cause staining which cannot be removed. It is recommended that you use the probe cover when carrying out training.
- Take care not to hit the probe against the wrist model. The wrist model is rigid, and damage to the probe element may result.
- Subcutaneous monitoring by means of ultrasound scanning can only be carried out on the puncture pad. It cannot be carried out on the wrist model.

## ■ Putting away


- 1 Turn the dial switch to the left to turn off the power.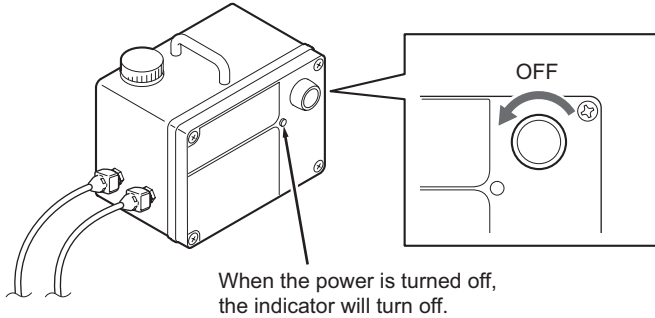


- 2 Set the tank with integrated pump so that the surface with the label is facing upward, and then turn on the power.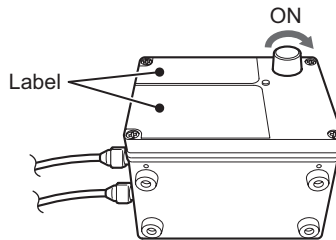


The water inside the blood vessel connection tubes will be returned to the tank with integrated pump.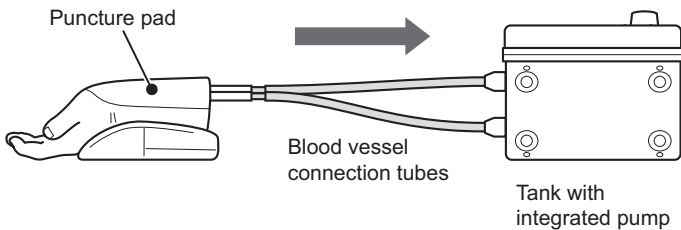

- 3** When all water has been removed from the blood vessel tubes, turn off the power.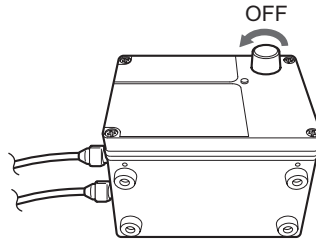


- 4** Disconnect the blood vessel connection tubes from the tank with integrated pump.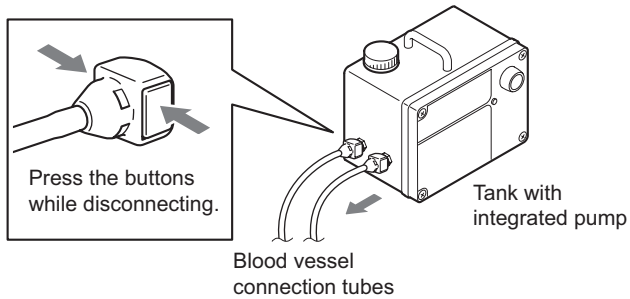

**CAUTION**

When disconnecting the blood vessel connection tubes, press the buttons to unlock the connectors and then remove the tubes. If you try to pull out the blood vessel connection tubes by force, they will become damaged and it will cause water to leak out.

- 5** Open the cap of the tank with integrated pump and pour out all of the water.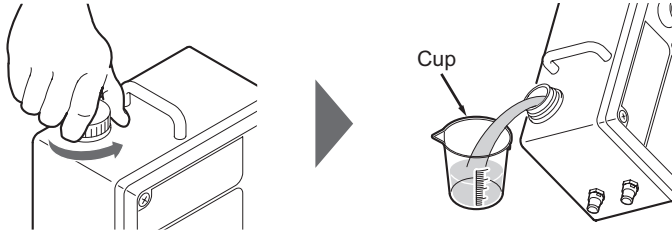


- 6** Disconnect the blood vessel connection tubes and the blood vessel tube from the puncture pad.

### < Replacing the blood vessel tube >

The blood vessel tube will deteriorate when it has been punctured many times. If this happens, replace the blood vessel tube with a replacement blood vessel tube which is provided as an accessory or sold separately.

- 1** Pull out the blood vessel tube from the puncture pad.

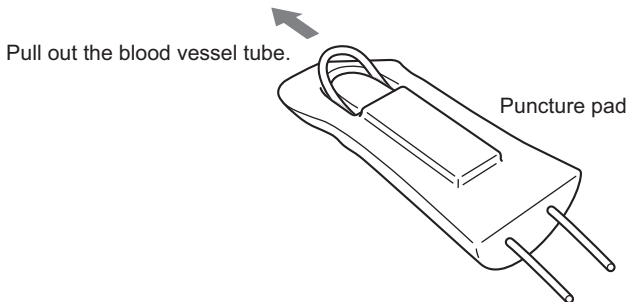- 2** From this point on, carry out steps **1** to **3** (in that order) in "Preparation".

# 7 Cleaning and storage

Be sure to observe the following when cleaning and storing the product.

- **If not using the product, disconnect the AC adapter from the wall outlet.**

When not using the product or when cleaning or storing it away, be sure to turn off the power switch and disconnect the AC adapter from the wall outlet.

- **Clean the wrist model and the surface of the puncture pad by wiping them gently with a soft cloth.**

- If it is extremely dirty, wipe it with a cloth which has been moistened with diluted alcohol, and then wipe it again with a dry cloth.
- Do not use benzine, thinner or other solvents. Otherwise, it may damage the product or cause the epidermis to deteriorate.
- If you write on the product using a ballpoint pen or similar object, the surface of the product will become smeared with ink which cannot be removed.

- **Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.**

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, problems with operation may result.

- **The puncture pad is a consumable part.**

The puncture pad will deteriorate when it has been punctured many times. In such cases, replace the puncture pad with a new one (replacement part) which can be purchased separately.

- **The blood vessel tube is consumable items.**

If carrying out repeated punctures and a large number of holes are made in the blood vessel tube, purchase a replacement blood vessel tube which is sold separately.

# 8 Troubleshooting

No.	Symptom or problem	Remedy	Reference page
1	The product does not operate when the power switch is turned on.	Check that the AC adapter is connected correctly to the tank with integrated pump and to the wall outlet.	"How to use" (P.35)
2	Backflow does not occur.	Check that the blood vessel tube and blood vessel connection tubes are not bent and that nothing is sitting on top of them.	"How to use" (P.36)
		Check that the blood vessel connection tubes are connected correctly and that they are securely installed.	"How to use" (P.31 ~ 33)
		Check that the cap of the tank with integrated pump is securely tightened.	"How to use" (P.34)
3	Water leaks out from the cap of the tank with integrated pump.	Check that the cap is securely tightened.	"How to use" (P.34)
4	There is a disagreeable odor.	Stop using the product and contact Sakamoto Model. (The inside of the product may be corroded.)	-

If any problems are not fixed when the remedies given above are carried out, contact Sakamoto Model.

# 9 Specifications

## ■ Overall product

Power supply	AC 100 – 240 V 50/60 Hz
Power consumption	5.1 W
Weight	1.5 kg
Continuous operating time	3 hours or less recommended

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## SAKAMOTO MODEL CORPORATION

Overseas Sales Department  
1-9-29, Senbahigashi, Minoh City,  
Osaka 562-0035, JAPAN

Tel +81-72-729-8350

Fax +81-72-729-8385

URL <https://www.sakamoto-model.com/>

E-Mail [trade@sakamoto-model.com](mailto:trade@sakamoto-model.com)

Product specifications and appearance are subject  
to change for improvement without notice.

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- Some of the components used in this product are manufactured by other companies, and product names, etc. for these components may be trademarks or registered trademarks of the manufacturers.