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Z102 Sakamoto microchip insertion trainer

【User's Manual】

Thank you for purchasing our product, “Sakamoto microchip insertion trainer”.

Read this user's manual carefully in order to use the product for a long time appropriately.


This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

【Cautions on Use】

- 1 Be careful with ballpoint pens and magic markers because their marks cannot be removed.
- 2 Take extra care when handling the product because it is fragile and will break if dropped.
- 3 If a defect or problem with the product is discovered during use, stop using the product immediately and contact Sakamoto Model or an authorized dealer.
- 4 This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee the same results during actual treatment. Please note that the specifications of this product may be changed without prior notice in order to make improvements to the product.

【Storage Instructions】

- 1 If the product gets dirty, use a soft cloth to remove the dirt or stain. For severe stains, use a neutral detergent diluted with water. Wipe it off using a cloth dampened with a disinfectant or alcohol and finish with a dry cloth. Do not use solvents such as benzine, thinner.
- 2 Avoid storing the product in direct sunlight or in locations with high temperature and humidity.
- 3 If there are many needle marks on the epidermis and it is interfering with training, please purchase a replacement epidermis and replace it as shown in the image below.

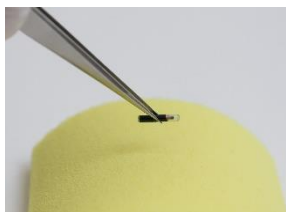Dummy microchip

Muscle sponge set

Attaching or replacing
the epidermis

URL <https://sakamoto-model.com>

E-Mail trade@sakamoto-model.com