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User's Manual (200611)

— I N D E X —

●Introduction/Table of Contents	1
●Purpose/Content	2
●Precautions Concerning Handling	3
●Usage	4-9
●Storage Method	9

!  
Note

Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damage or with parts missing, please contact our company or the distributor.

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Scan this QRcode with your device to see our homepage.

## Introduction

---

Thank you for purchasing our product, “Female/Male Catheterization • Enema Simulator Plus”.

Read this user’s manuals carefully in order to use the product for long time and appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon and check if anything on the main unit or accessories are missing or broken.

## Table of Contents

---

Introduction/ Table of Contents.....	1
Purpose/ Content.....	2
Precautions concerning Handling.....	3
Usage.....	4 - 9
Storage Method.....	9


## Purpose

Training performed by the following items:

1. Intermittent Urethral Catheterization
2. Continuous Urethral Catheterization
3. Confirmation of Indwelling Balloon Catheter
4. Genital Wash
5. Glycerin Enema Method
6. Cleaning around the anus

## Content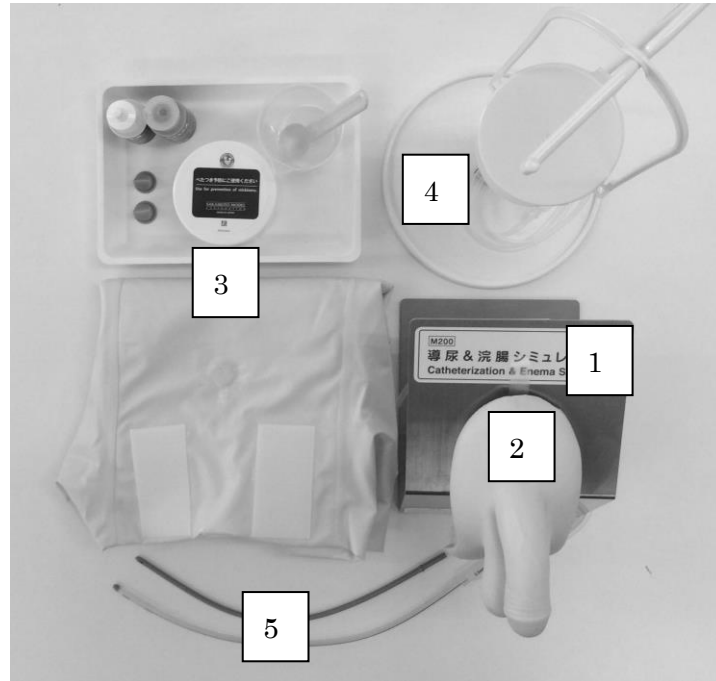

※This picture is Male Catheter・Enema Simulator.

- |  |                 |
|--|-----------------|
| 1. Cradle (For table use)  | 1 piece         |
| 2. Genitalia for Catheter and Enema  | 1 piece         |
| 3. Silicon oil, Baby powder, Simulated stool,<br>Cup for mixing simulated stool, Waterproof pants,<br>Muddler Tray, Parts for mounting<br>Backflow prevention valves | 1 each<br>2 set |
| 4. Irrigator bottle and stand  | 1 each          |
| 5. Catheters(8 EG Nelaton Catheter, 16 FR Balloon Catheter)  | 1 each          |


## Cautions Concerning Handling

1. If resin materials in close contacts with the main unit or an external genitalia for a prolonged, the product might be deformed or discolored. Treat the product carefully.
2. Do not use any staining solution or inks on the model to not to be absorbed in to the skin.
3. Do not disassemble the Genitalia parts. The Genitalia and the base are attached by glue.
4. When carry the Genitalia parts, please hold the base.
5. When you remove bottles, please take away tube first. Then remove bottles.  
Do not fasten bottles tightly when attaching.  
L-shaped connector part on the bottle can not be removed.  
If forcibly removed, water leakage or malfunction will occur.
6. Be sure to apply the lubricant oil which provided. Please use only the supplied lubricant oil to prevention of breakage.
7. Please deflate a balloon catheter BEFORE pull out.
8. Wrap the simulated stool in paper, and dispose with the combustible waste.  
  
Do not dispose into drains.
9. This waterproof pants are not completely waterproofed. it may penetrate from the seam dueto pressure or deterioration.
10. If a defect or probrem is found during use. Stop using and contact to Sakamoto Model or authorized dealer.
11. This product is a simulator capable of supporting advanced medical training.  
  
However Sakamoto Model do not guarantee the same results during actual treatment.
12. Please note that the specifications of this product may be changed without prior notice in to make improvements.

# Instructions for use

## ■ Setting bottles


1 Attach the backflow prevention valve.

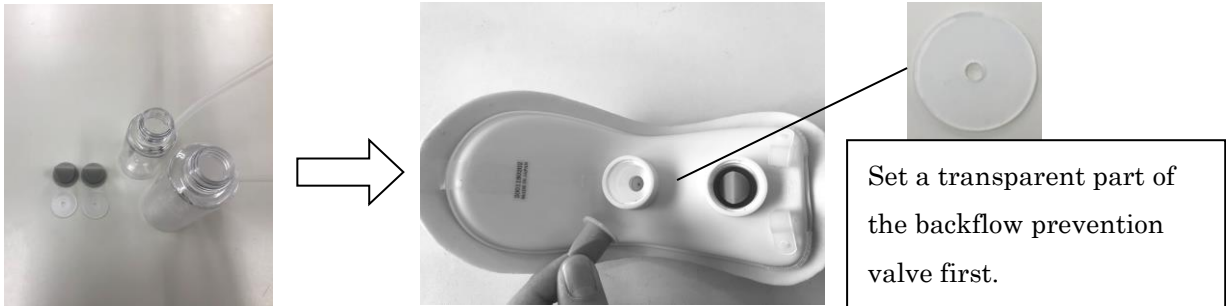2 Set bottles for urethral catheter and enema.


3 Insert tubes on fixing part.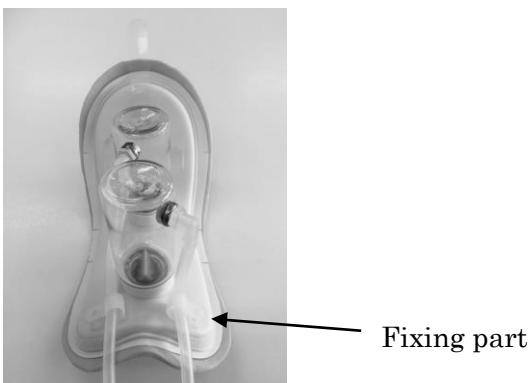

## ■ Preparation—Common


- 1 Confirm both of bottles on correct place.

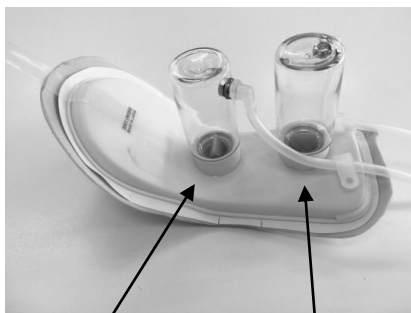Bottle for urethral catheter  
(with white connector)

Bottle for enema  
(with yellow connector)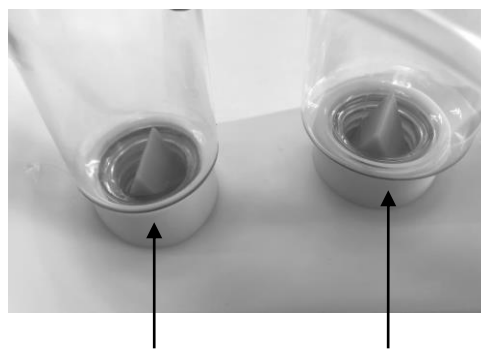


Check the valve are set.

※Be careful not to forget put the packing.


## ■ Preparation—Catheterization

- 1 Set the genitalia parts on the cradle.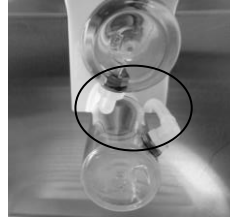


※Back side photo. Be careful not to bend the tubes.

- 2 Connect the tube from genitalia part and tube from irrigator bottle.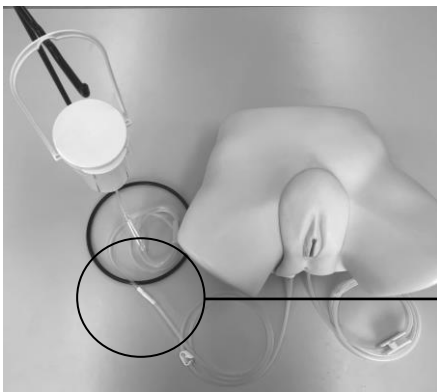

- 3 Add 500cc of water into irrigator bottle and open 2 clamps (Irrigator side and genitalia side).

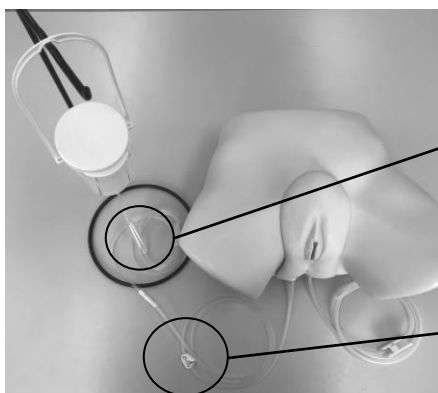## POINT

- Before insertion, please apply the special lubricant at the tip of the Nelaton catheter. (If the catheter inserted without applying lubricant, it will rub and damage in the urethra. In addition, do not apply excessive force because if the insertion angle is incorrect, the catheter cannot be inserted.)
- Once the Nelaton catheter reaches the inside of the bladder (refer to figure on below), water collects inside the bladder and then discharges. Therefore, prepare a tray to collect the discharged water from the catheter. (Refill water inside the irrigator as necessary.)

- 4 Preparation completed. You can start the practice.  
※Catheters can be used 12Fr to 18Fr.

## ■ Preparation—Enema

- 1 Please prepare an enema drainage bucket. (Drainage bucket is not included.)

Open the clamp and begin training.


※Appropriate size of enemas is 12Fr to 18Fr (4mm to 6mm).

※Resistance to the enema fluid can be adjusted by opening/closing the clamp.

**Note—Apply the dedicated lubricant oil to the enema tool before starting training.**

## ■ Preparation—Attaching


- 1 Attach the velcro to the catheterization and enema bottles.

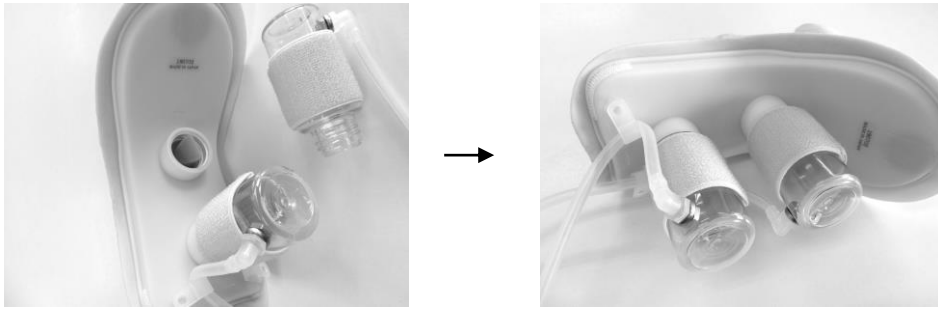- 2 Wear the waterproof pants.
- 3 Attach the upper part of genitalia to the magnet of the waterproof pants.  
(There is a magnet inside of the top)  
Attach the Velcro of genitalia to the velcro on the waterproof pants.

- 4 Set the bottle on the illigator stand.  
Connect the tube of the bottle for urethral catheter and the tube of the illigator bottle.  
Put the tube of the bottle for enema in the drainage container.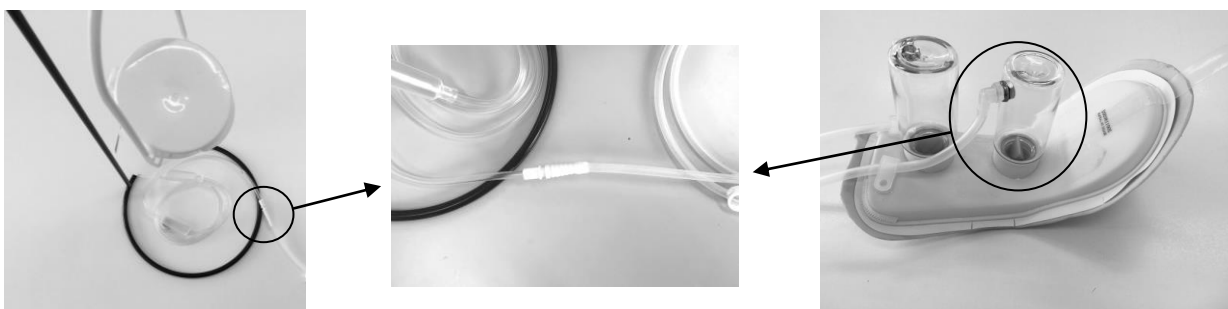

※Bottle for urethral catheter


### POINT

Waterproof pants are not completely waterproof.  
Attach a genitalia part after wearing waterproof pants.


If wear the waterproof pants after attaching the genitalia, it way fall or be damaged.  
Place the stand on a stable surface.

## ■ Storage Instructions for catheter

- 1 Have a tray ready to collect the water that is discharged.
- 2 Remove genitalia parts from main unit.
- 3 Close clumps and remove tubes from bladder part and Irrigator.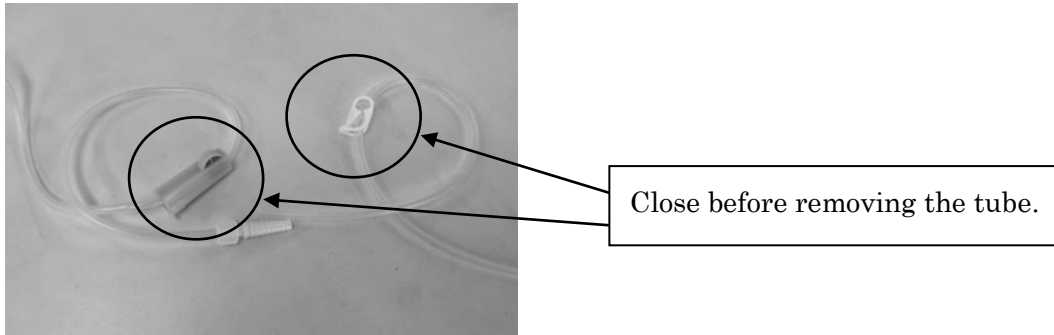

- 4 Remove tube from fixed part.  
Detach bottle for bladder and open clump for draining.

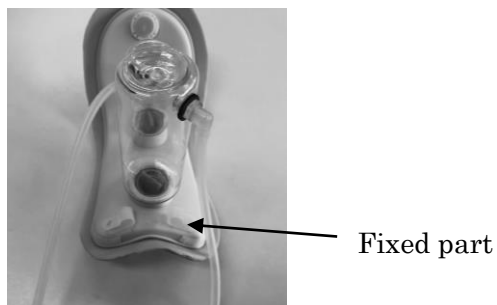※Please be care to be lost backflow prevention valves when you detach bottles from genitalia part.

- 5 Wipe water and dry before reset genitalia part.  
(about how to reset genitalia part, please refer “preparation-catheter” .)

## ■ Storage for enema

- 1 Remove genitalia part from main unit.
- 2 Close the clamp of tube connecting with bottle.
- 3 Detach bottle and open clump for draining.

※Please be care to be lost backflow prevention valves when you detach bottles from genitalia part.

- 4 Wipe water and dry before reset genitalia part.  
(About how to reset genitalia part, please refer “preparation-catheter” .)

## ※Points of Genital Washing

- 1 Perform training using water for the main unit.  
When water is used, the surface may become sticky or not so smooth. This is no problem regarding quality; however, the surface becomes easy to tear. So, pay sufficient attention to handling.
- 2 Do not use any chemicals on the main unit. (Otherwise, the product might fail.)

## Care and Storage

Please check the following points for maintenance cleaning and storage.

- Wipe off lightly by soft cloth when you clean surface.

Wipe off by soft cloth with diluting detergent and alcohol with water when you find soiled place.

Do not use benzene, thinner and solvent. Do not spray alcohol directly. It might cause of damage and degradation.

- To prevent degradation and mold, please dry inside of bottle after finish your practice.

- Apply attached powder to protect genitalia part and prevent stickiness.

Apply attached powder if you do not use long time and feel stickiness.

- Do not store the product in a place exposed to direct sunlight or hot and humid air.

- Put this model in storage bag and store it.

---

**SAKAMOTO MODEL**  
C O R P O R A T I O N

---

Head office

34,Higashitakagicho,Shimogamo  
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka  
562-0035,JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com

---