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
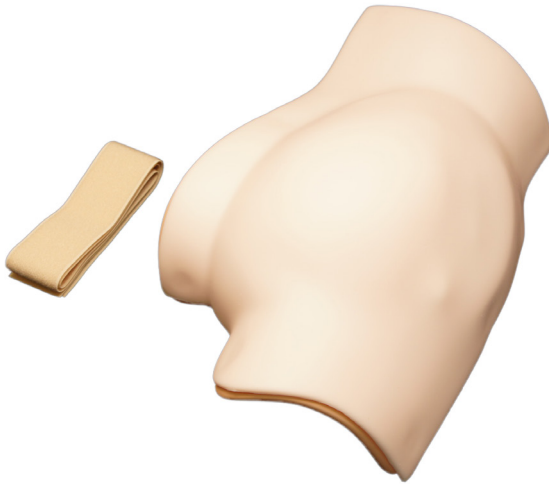
M153-2 Buttocks Intramuscular Injection Simulator (wearing type)

Buttockmate Right

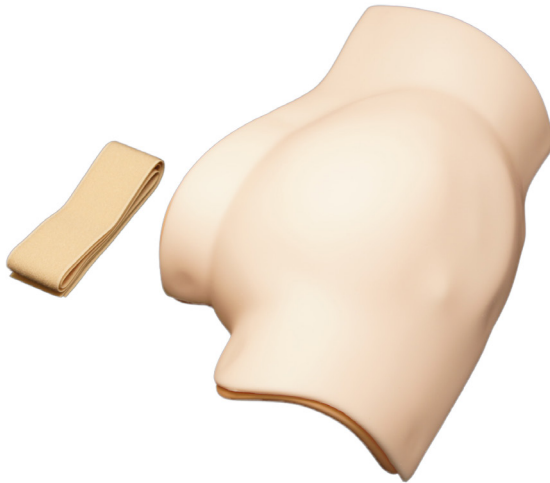User's Manual

Only use water* for carrying out injection training using this product!

Furthermore, do not use syringes with bent needle tips!

* Use either tap water or distilled water for injection use.

Table of contents

1	Introduction	13	5	How to use	17
2	Product contents	13	6	Cleaning and storage	20
3	Cautions on use	14	7	Troubleshooting	21
4	Quick Preparation Guide	15	8	Specifications	22

You can view this product's introductory page on our company's website by using either the following URL or the QR code* at right. The page includes a video that you can watch which shows how to use the product.

● Sakamoto Model Corporation's Introductory page for the "Buttockmate Right"
https://www.sakamoto-model.com/product/injection/m153_2/

* QR code is a registered trademark of Denso Wave Inc.

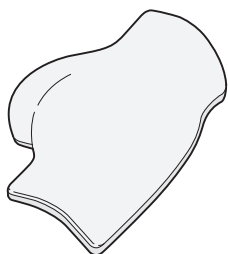
1 Introduction

Thank you very much for purchasing the "Buttockmate Right".


This user's manual contains cautions and notes on how to use this product safely. Please read this manual carefully in order to ensure that you know how to use the product safely. In addition, after reading the manual, keep it in a safe place where it can be referred to at any time later on.

After purchasing the product, check the package contents carefully to make sure that nothing is missing or broken. If you notice any problems, contact Sakamoto Model.

2 Product contents


Main unit X 1

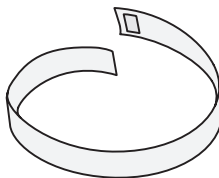Wearable belt X 1

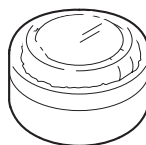
AAA-size batteries X 2

Baby powder X 1

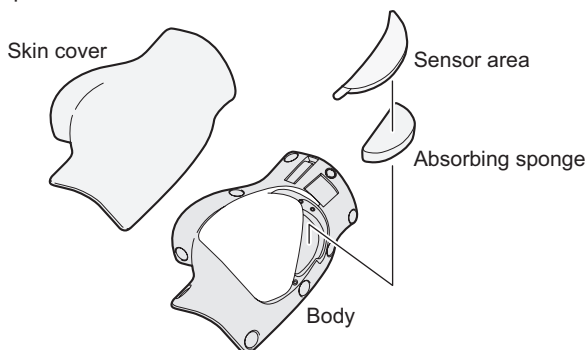Main unit structural parts

3 Cautions on use

In order to use this product safely, be sure to observe the following.

(Items which are particularly important are marked with " " .)

● **Handle these items gently.**

- Do not touch them with sharp objects such as knives, scissors or needles.
(The injection area can be pierced with the injection needle. In addition, inserting the needle into the muscle area will not cause any problems with use.)
- Do not pull or twist them with a strong force.
- Do not subject them to severe shocks.

● **Always use fresh needle tips in the syringes when practising.**

If you use a syringe with a needle that has a bent or damaged tip, it may cause the inside of the main unit to become damaged or the epidermis to become degraded.

● **Only use water as the fluid for injecting.**

If a fluid other than water is used, damage or disagreeable odors may result. Use either tap water or distilled water for injection use.

● **Do not inject anywhere other than inside the injection area.**

The area where injection is possible is limited by the internal structure.

If you inject anywhere outside the injection area or the muscle area, it may damage the product. (Refer to "4 Quick Preparation Guide" (P.15 - 16))

● **If you notice any problems when using the product, stop using it.**

If problems occur with the product during use, stop using it, remove the batteries, and contact Sakamoto Model. If you continue using the product, accidents may result.

● **Do not write on the skin cover with items such as ballpoint pens.**

If you write on the skin cover using a ballpoint pen or similar object, the surface will become smeared with ink which cannot be removed.

● **Do not disassemble or modify the product.**

Do not disassemble any part of the product other than the parts which are removed during normal use. Furthermore, do not modify the product.

The product may become damaged or problems with operation may result.

● **Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.**

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, then problems with operation or discoloration or deterioration may occur.

● **Do not use this product for other applications.**

Do not use this product for applications other than the original application for which it is designed. The product may become damaged or injury may result.

● **There are limits to the effectiveness of simulator training.**

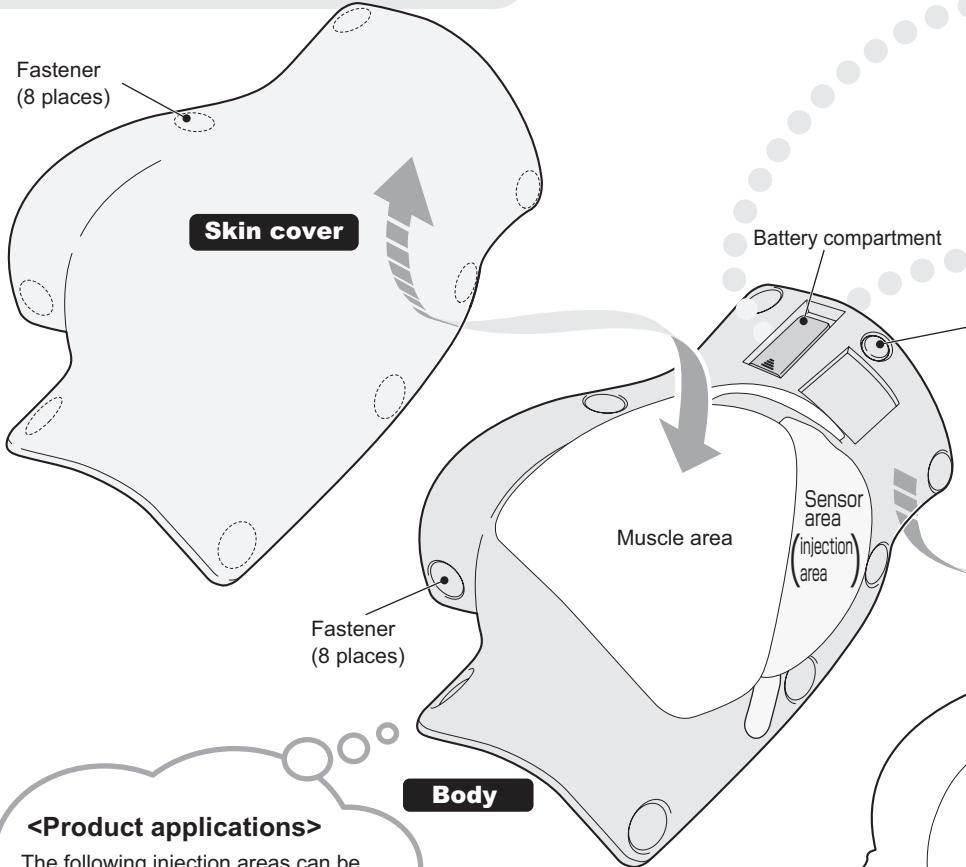
This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee that training using this product will provide the same level of expertise as actual treatment.

4 Quick Preparation Guide

This guide describes the mechanism and how to start using the "Buttockmate Right".
Be sure to read the "Cautions on use" (P.14) before using this product.

1

Remove the skin cover from the body.

<Product applications>

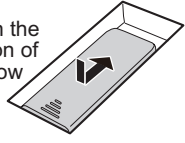
The following injection areas can be checked and injection training carried out for them!

- Clarke's points
- Checking of upper outer quadrant of buttocks
- Hochstetter's site



2

Open the battery compartment cover, insert the accessory batteries, and then close the cover.


Slide in the direction of the arrow

Battery compartment cover

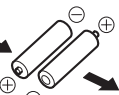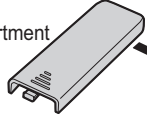AAA-size batteries (2 pcs.)

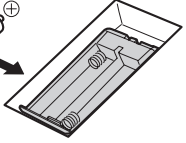
CAUTION

Insert the batteries so that the polarities are correct.

3


Press the power button on the underside of the body to turn on the power.

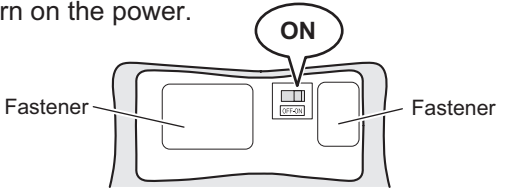Speaker

4

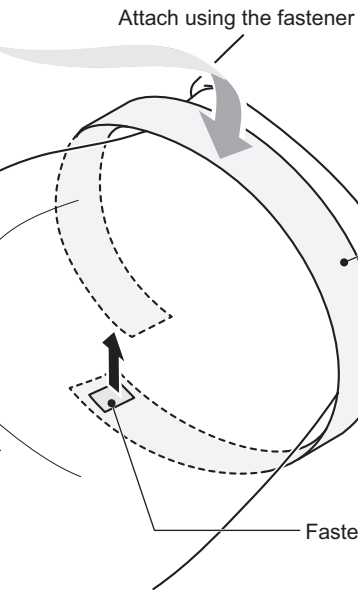Place the skin cover back onto the body.

5

Wrap the wearable belt around the hips of the wearer and then attach the "Buttockmate Right" .

CAUTION

- The product can be worn in either the prone or the lateral position.

5 How to use

■ Preparation for training

Prepare the following items.

- Syringe.....Injection needles with a gauge of between 21G and 25G can be used.
 - * Between 21G and 23G is recommended.
 - * Do not use needles which are 20G or less (0.9 mm or more).
- Injection liquid....Water (either tap water or distilled water for injection use)

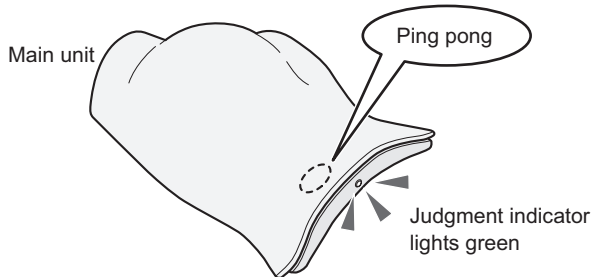
■ Training (product operation)

Check that the product is attached correctly while referring to "4 Quick Preparation Guide" (P. 15), and then carry out training.

This product can be used for checking the following injection areas and carrying out injection training for them.

- Clarke's points
- Checking of upper outer quadrant of buttocks
- Hochstetter's site

When injection is carried out at the correct position and the correct depth, the judgment indicator lights and the "ping pong" tone of the chime sounds.

⚠ CAUTION

- The wearer should not stand upright while putting on this product. The product may fall down and become damaged or injury may result.
- Do not inject in areas outside the injection area or the muscle area. (Refer to "4 Quick Preparation Guide" (P.15))
- The injection liquid (water) can only be injected while the judgment indicator and the chime are operating.
- If a large amount of water is injected into the injection area, it may leak out. Remove the absorbing sponge frequently and wring out the water. (Refer to "■ Putting away and replacing the absorbing sponge". (P.18))

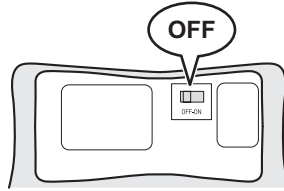
● TIP

The injection area can be disinfected with alcohol before injecting.


■ Putting away and replacing the absorbing sponge

When putting away the absorbing sponge or replacing it with a new one because it has become damaged, remove it by following the procedure given below.


- 1 Turn off the power switch.

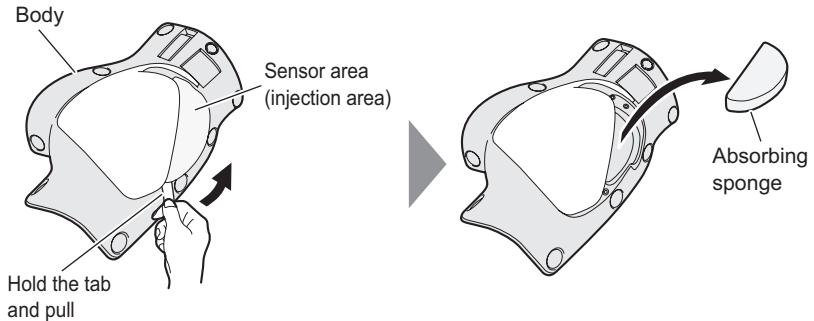- 2 Remove the skin cover and sensor area, and then take out the absorbing sponge.

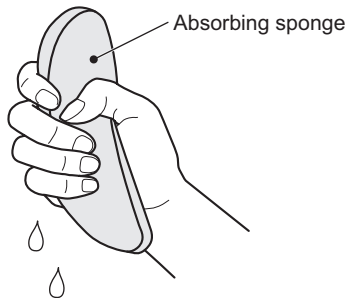
- 3 <If reusing the absorbing sponge>
Wring out the absorbing sponge and then let it dry.


<If replacing the absorbing sponge>
Have the new absorbing sponge ready.

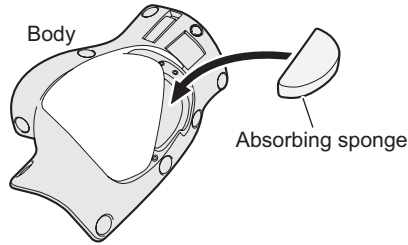4 Insert the absorbing sponge into the body.

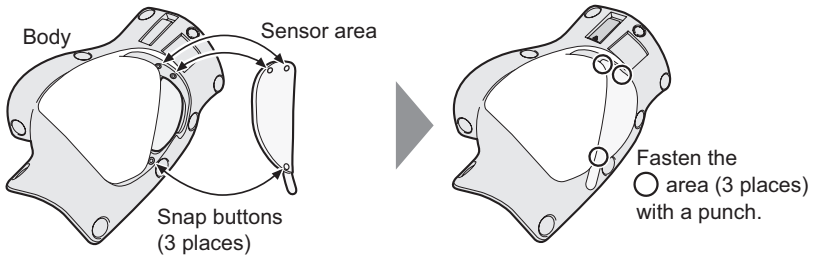5 Install the sensor area to the body, and then fasten the snap buttons (3 places).

TIP

Securely fasten the snap buttons when installing the sensor area.
If the snap buttons are not securely fastened in all three places, the judgment indicator and chime will not operate.

6 Install the skin cover.

7 Follow the procedures in "Cleaning and storage" before putting the unit away.

6 Cleaning and storage

Be sure to observe the following when cleaning and storing the product.

● **Dry out the inside of the product before storing it.**

Thoroughly wipe off the water from all parts, and then let the product stand for about half a day to a day to allow the inside to fully dry out.

If the product is stored away without fully drying it first, disagreeable odors will result.

● **To clean the main unit, use a soft cloth to gently wipe the surface.**

- If they are extremely dirty, wipe them with a cloth which has been moistened with diluted detergent or alcohol, and then wipe them again with a dry cloth.
- Do not use benzine, thinner or other solvents. Otherwise, it may damage the product or cause the surface to deteriorate.
- If you write on the skin cover using a ballpoint pen or similar object, the surface will become smeared with ink which cannot be removed.

● **Remove the batteries when not using the product for a long period.**

If the batteries leak inside the battery compartment, problems with operation will result.

● **Coat the skin cover with baby powder to protect it and to stop it from becoming sticky.**

If not using the product for a long period of time or if the skin cover is sticky, coat it with the accessory baby powder.

● **Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.**

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, then problems with operation or discoloration or deterioration may occur.

● **The skin cover, sensor area and absorbing sponge are consumable parts.**

These parts will deteriorate when the product has been injected many times. In such cases, replace the respective parts with replacement parts which can be purchased separately.

7 Troubleshooting

No.	Symptom or problem	Remedy	Reference page
1	Judgment indicator does not light.	<ul style="list-style-type: none"> • Check that the batteries are inserted the correct way. (There may be no batteries inserted or the polarities may be incorrect.) • Check that the sensor area is installed correctly. (The snap buttons may have become detached.) 	[4 Quick Preparation Guide] (P.15 - 16) [5 How to use] (P.19)
2	Chime does not sound.	<ul style="list-style-type: none"> • Replace the batteries with fresh ones. (The batteries may be nearly spent.) 	[4 Quick Preparation Guide] (P.15 - 16) [5 How to use] (P.19)
3	Chime is very quiet.	Replace the batteries with fresh ones. (The batteries may be nearly spent.)	[4 Quick Preparation Guide] (P.15 - 16)
4	Chime sounds when the power is turned on.	Replace the sensor area. (The internal sensors may be faulty.)	[5 How to use] (P.18 - 19)
5	Judgment indicator lights when the power is turned on.		[5 How to use] (P.18 - 19)
6	There is a disagreeable odor.	Stop using the product and contact Sakamoto Model. (The inside of the main unit may be corroded.)	—

If any problems are not fixed when the remedies given above are carried out, contact Sakamoto Model.

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SAKAMOTO MODEL

C O R P O R A T I O N

Overseas Sales Department
1-9-29, Senbahigashi, Minoh City,
Osaka 562-0035, JAPAN

Tel +81-72-729-8350

Fax +81-72-729-8385

URL <https://www.sakamoto-model.com/>

E-Mail trade@sakamoto-model.com

Product specifications and appearance are subject
to change for improvement without notice.

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- Some of the components used in this product are manufactured by other companies, and product names, etc. for these components may be trademarks or registered trademarks of the manufacturers.