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

M203 Intravenous Injection Trainer

Sakamoto electric venipuncture trainer

User's Manual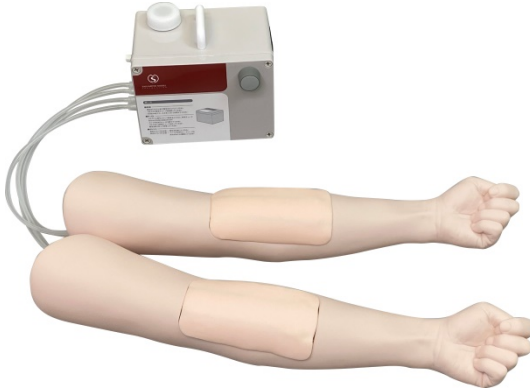

Only use **water**^{*} or **artificial blood** for carrying out injection training using this product!
Furthermore, do not use syringes with bent needle tips!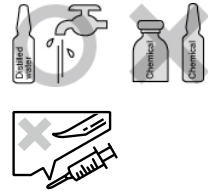

* Use either tap water or distilled water for injection use.

Table of contents

| | | | | | |
|---|----------------------|----|---|----------------------|----|
| 1 | Introduction | 25 | 6 | How to use | 31 |
| 2 | Product contents | 26 | 7 | Cleaning and storage | 43 |
| 3 | Cautions on use | 27 | 8 | Troubleshooting | 44 |
| 4 | Product applications | 29 | 9 | Specifications | 46 |
| 5 | Part names | 30 | | | |

1 Introduction


Thank you very much for purchasing the " Sakamoto electric venipuncture trainer ".

This user's manual contains cautions and notes on how to use this product safely. Please read this manual carefully in order to ensure that you know how to use the product safely.

In addition, after reading the manual, keep it in a safe place where it can be referred to at any time later on.

After purchasing the product, check the package contents carefully to make sure that nothing is missing or broken. If you notice any problems, contact Sakamoto Model.


2 Product contents

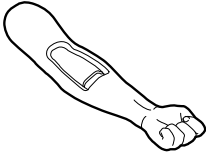
Arm model ×2


Injection pad ×2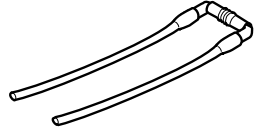


Blood vessel tubes ×2

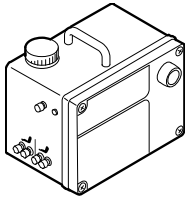
Tank with integrated pump ×1

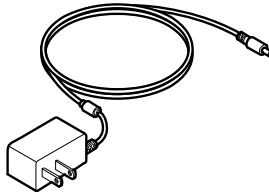AC adapter ×1

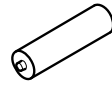
AA-size batteries ×6

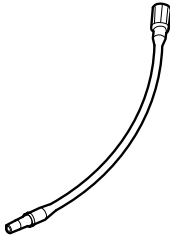
Blood vessel connection tube ×4

Cup ×1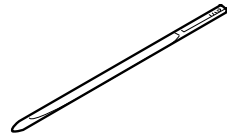


Blood vessel threader ×1

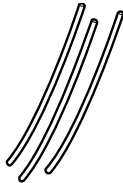Replacement blood vessels
(standard size) ×10

Spare stopper cap ×2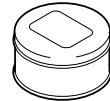

Baby powder ×1

3 Cautions on use

In order to use this product safely, be sure to observe the following.

(Items which are particularly important are marked with "●".)

● **Do not subject the product to impacts or other forces.**

- Do not drop.
- Do not bump against solid objects.
- Do not apply excessive loads.

If any of the above occur, damage may result.

● **Handle the injection pad gently.**

- Do not touch it with sharp objects such as knives, scissors.
- Do not pull or twist them with a strong force.

● **Always use fresh needle tips in the syringes when practising.**

If you use a syringe with a needle that has a bent or damaged tip, it may cause the blood vessels to become damaged or the injection pad to become degraded.

● **Only use water or artificial blood as the fluid for injecting.**

If a fluid other than those mentioned above is used, damage or disagreeable odors may result. Use either tap water or distilled water for injection use. To create artificial blood, add 1 to 2 drops of the artificial blood concentrate to water.

● **Do not disassemble or modify the product.**

If you attempt to disassemble or modify the product, it may become damaged or problems with operation may result. Never disassemble or modify the product.

● **Do not write on the arm model with items such as ballpoint pens.**

If you write on the arm model using a ballpoint pen or similar object, the surface of the arm model will become smeared with ink which cannot be removed.

● **Do not use continuously for long periods.**

As a means to protect the product, avoid using it for long periods of time. (No more than 3 hours at a time is recommended.)

● **Hold the body of the AC adapter when removing it from an outlet.**

When disconnecting the AC adapter from the wall outlet, hold it by the AC adapter body, not by the cable. If you hold by the cable when disconnecting it, the cable may become damaged, and it may lead to electric shocks or fire occurring. In addition, do not connect or disconnect the cable with wet hands.

- **If you notice any problems when using the product, stop using it.**

If problems occur with the product during use, stop using it, turn off the power, and disconnect the power cable from the electrical outlet. If the batteries have been used, remove them. Then contact Sakamoto Model. If you continue using the product, accidents may result.

- **Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.**

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, problems with operation may result.

- **There are limits to the effectiveness of simulator training.**

This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee that training using this product will provide the same level of expertise as actual treatment.

- **Do not use this product for other applications.**

Do not use this product for applications other than the original application for which it is designed. The product may become damaged or injury may result.

4 Product applications

This product can be used to carry out the following training.



● Intravenous injections

● Drawing blood


- The pump creates the effect of blood circulating through the veins, so that backflow can be checked when drawing blood.

5 Part names

■ Arm model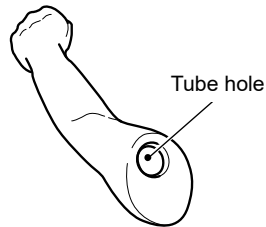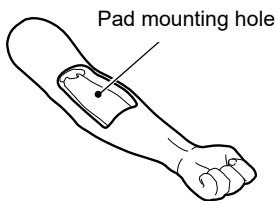

■ Tank with integrated pump

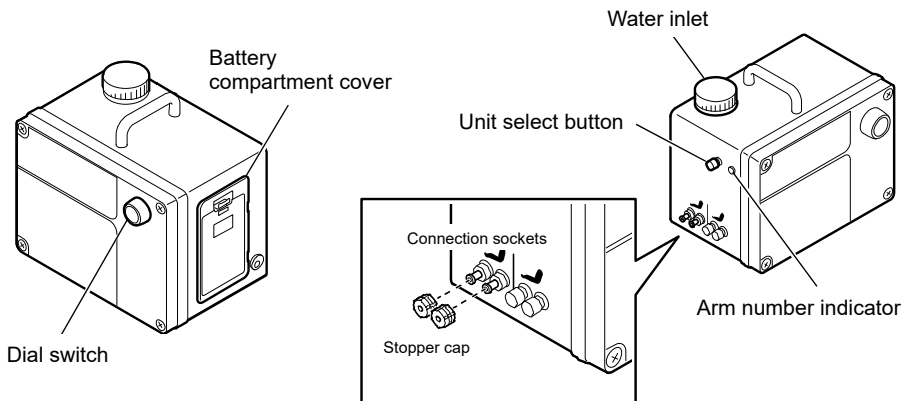6 How to use


■ Preparation

● TIP


This product includes two arm models which can both be connected at the same time so that two people can carry out training simultaneously. The following describes how to connect a single arm model, but the connection procedure is the same for both.

- 1 Insert the batteries into the tank with integrated pump, or connect the AC adapter.**

<When using the batteries>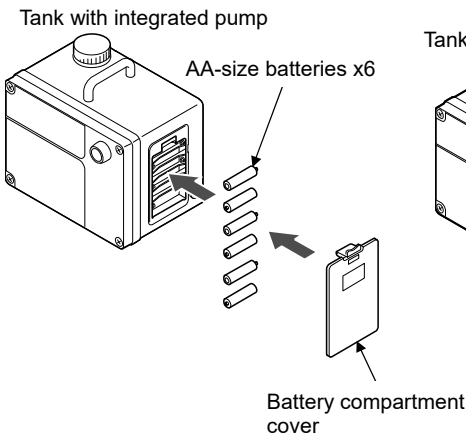


<When using the AC adapter>

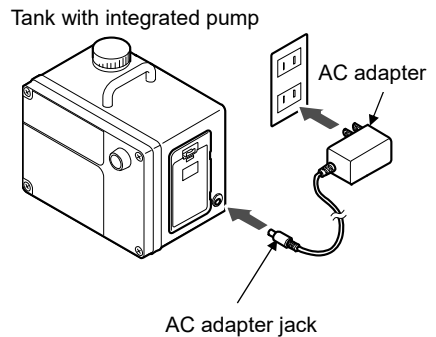⚠ CAUTION

- Be sure to insert the batteries so that their polarities are correct.
- When using the batteries (alkaline) to operate two arms, the batteries will last for approximately 6 hours.
- Do not handle the AC adapter with wet hands. Otherwise electric shocks may occur.


- 2** Pass the end of the blood vessel tube through the blood vessel threader.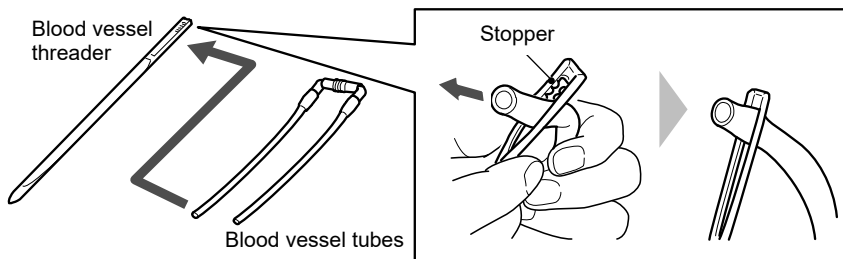

TIP

Widen the hole in the blood vessel threader, and insert the end of the blood vessel tube so that it hooks onto the stopper.


- 3** Pass the blood vessel threader into the through hole and pull the blood vessel tube through the pad.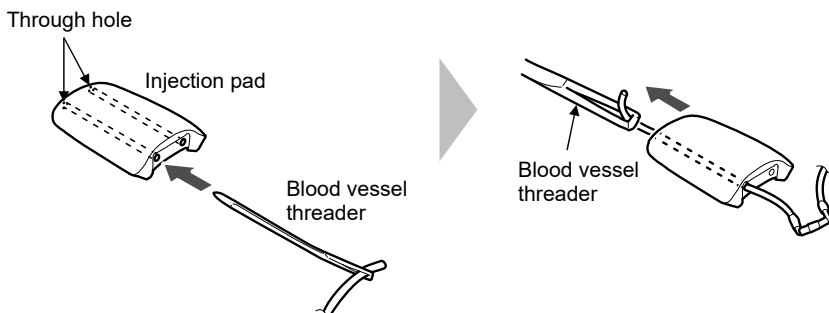

TIP

If the tube is difficult to pull through, apply baby powder to the tube.


- 4** Repeat steps 2 to 3 for the tube on the opposite side.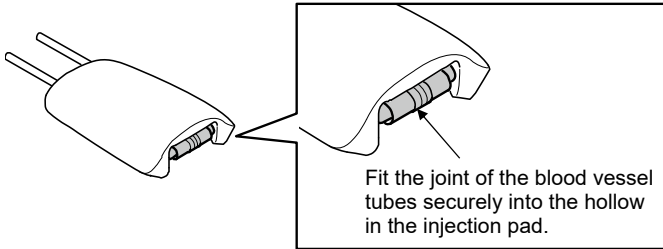


TIP

If the blood vessel tubes are pulled too strongly, the injection pad may become bent. If this happens, re-insert the blood vessel tube.


- 5** Insert the blood vessel connection tubes into the blood vessel tubes.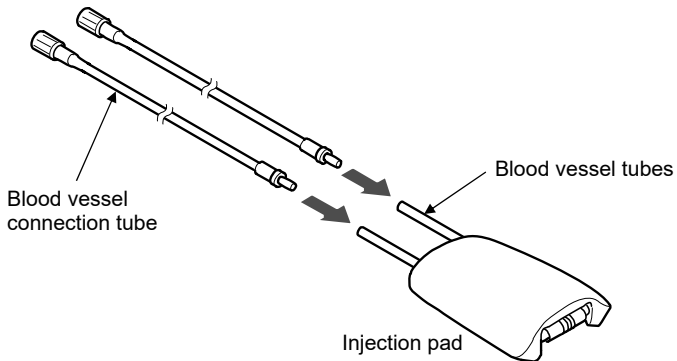

- 6** Pass the blood vessel connection tubes through the arm model, and fit the injection pad into the pad mounting hole.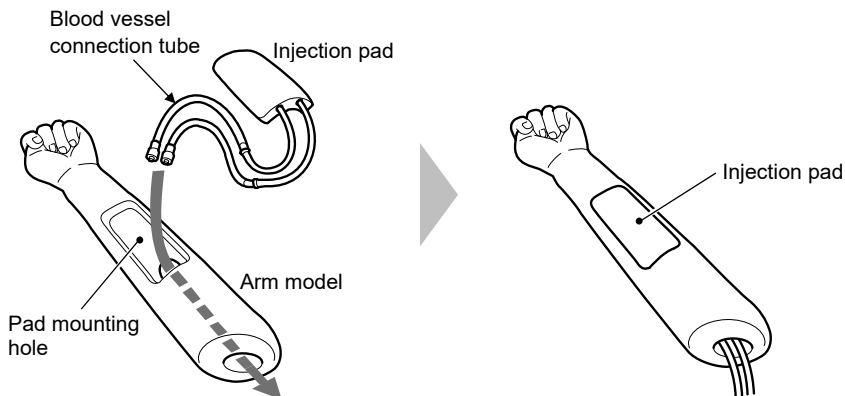

- 7** Turn the stopper cap to remove it from the tank with integrated pump.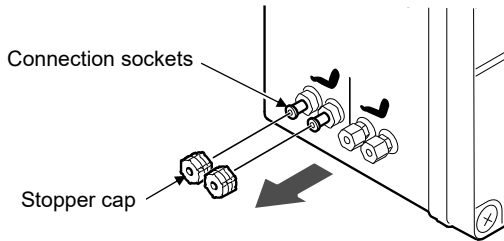


CAUTION

- Do not remove the stopper cap from the side which is not being used. Otherwise water will come out when the pump is operated.
- Be careful not to lose the stopper cap.

TIP

- Two sockets are used for each arm model.
- When using only a single arm model at a time, either the left or right set of sockets can be used.
- By using two arm models, two people can carry out training simultaneously.

- 8** Insert the blood vessel connection tubes into the connection sockets.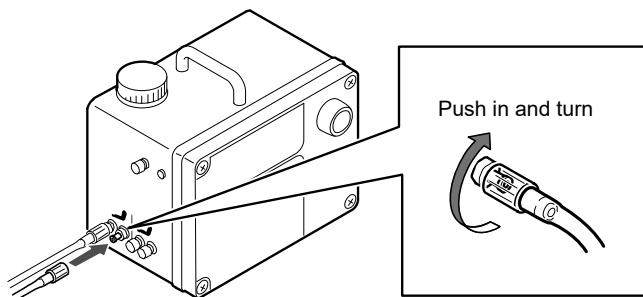

CAUTION

- Do not turn the connectors of the blood vessel connection tubes any further than necessary. The tube is made of plastic, and may become damaged.
- Always be sure to turn the blood vessel connection tubes counterclockwise to unlock them first before disconnecting them. If you keep pulling them, the connectors will break.
- Connect the blood vessel connection tubes carefully so that they do not break.


9 Pour 200 ml of water into the cup.

TIP

- Use either tap water or distilled water for injection use. Artificial blood can also be used. To create artificial blood, add 1 to 2 drops of the artificial blood concentrate to water.
- In the following procedure, the term "water" is used exclusively.

10 Pour the water into the water inlet of the tank with integrated pump, and then tighten the cap securely.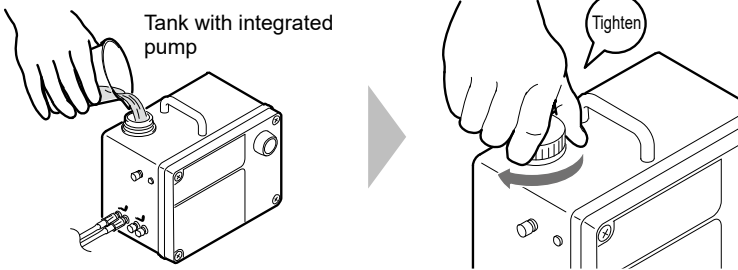


CAUTION

- Be careful not to spill the water. If the water gets spilled, wipe it away immediately. Otherwise electric shocks may occur.
- Tighten the cap securely. If the cap is not tightened securely, water may leak out.

■ Training

- 1 Turn the dial switch of the tank with integrated pump to the right to turn on the power.


Tank with integrated pump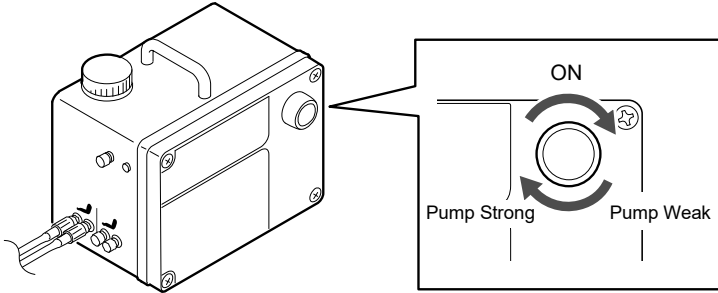

● TIP

The pump will make a whirring noise while it is operating, but this is normal and not the sign of a malfunction.

- 2 Press the unit select button to select the number of arm models being used.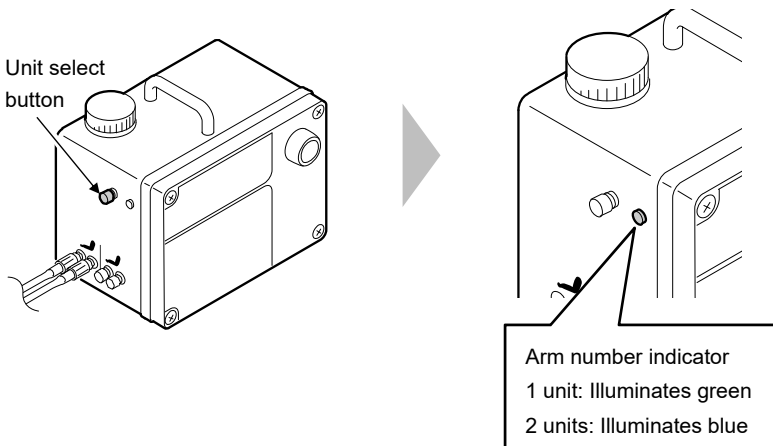


<Pump overload detection>

The tank with integrated pump is equipped with an overload detection function in order to prevent damage due to overloading.


If the blood vessel connection tubes become bent or pressed and cause flow to become impeded, the pump will stop operating and the arm number indicator will flash red.

In such cases, turn off the power of the tank with integrated pump, remove the cause of the problem and then turn the power back on.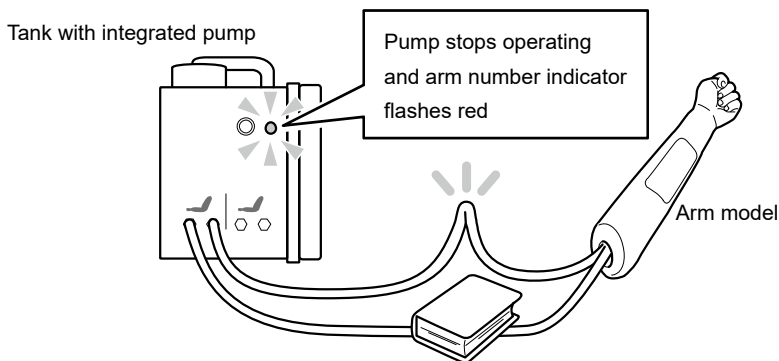


■ Putting away


- 1 Turn the dial switch to the left to turn off the power.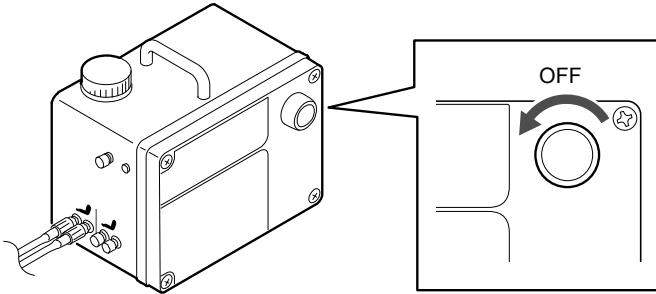


- 2 Set the tank with integrated pump so that the surface with the label is facing upward, and then turn on the power.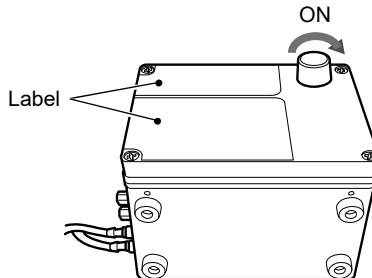


The water inside the blood vessel connection tubes will be returned to the tank with integrated pump.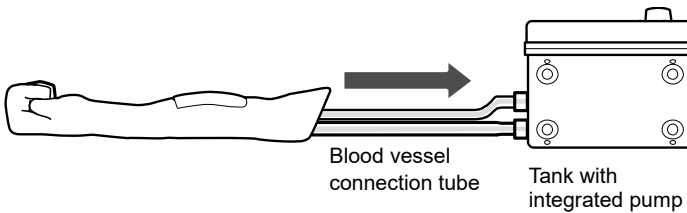

- 3** When all water has been removed from the blood vessel tubes, turn off the power.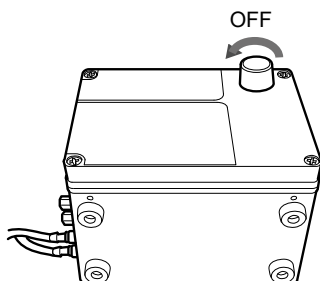

- 4** Open the cap of the tank with integrated pump and pour out all of the water.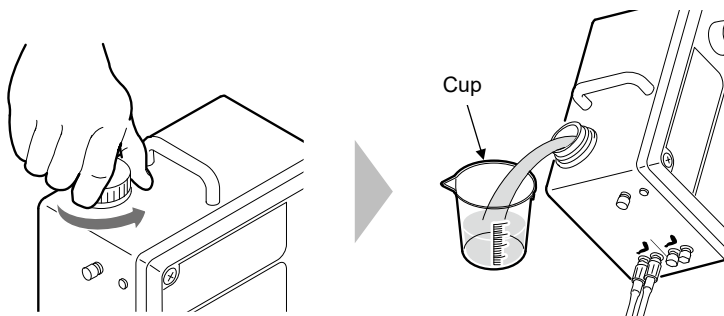

CAUTION


Open the cap and release the pressure inside the tank before pulling out the blood vessel connection tubes. If the blood vessel connection tubes are pulled out first, water will come out.

- 5** From this point on, carry out steps **8** to **1** (in that order) in "Preparation".

<Replacing the blood vessel tubes>

The blood vessel tubes will deteriorate when they have been injected many times.

If this happens, replace the blood vessels with replacement blood vessels (standard size) which are provided as accessories or sold separately.

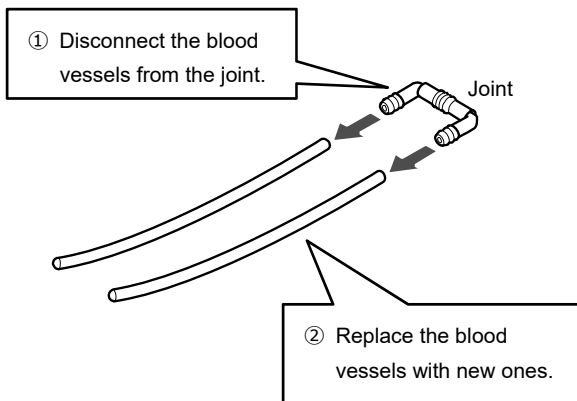7 Cleaning and storage

Be sure to observe the following when cleaning and storing the product.

- **Remove the batteries when not using the product.**

If the batteries leak inside the battery compartment, problems with operation will result.

- **If not using the product, disconnect the AC adapter from the wall outlet.**

When not using the product or when cleaning or storing it away, be sure to turn off the power switch and disconnect the AC adapter from the wall outlet.

- **Clean the arm model and the surface of the injection pad body by wiping them gently with a soft cloth.**

- If it is extremely dirty, wipe it with a cloth which has been moistened with diluted alcohol, and then wipe it again with a dry cloth.
- Do not use benzine, thinner or other solvents. Otherwise, it may damage the product or cause the epidermis to deteriorate.
- If you write on the product using a ballpoint pen or similar object, the surface of the product will become smeared with ink which cannot be removed.

- **Coat the injection pad with baby powder to protect it and to stop it from becoming sticky.**

If not using the product for a long period of time or if the injection pad is sticky, coat it with the accessory baby powder.

- **Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.**

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, problems with operation may result.

- **The injection pad is a consumable part.**

The injection pad will deteriorate when it has been injected many times. In such cases, replace the injection pad with a new one (replacement part) which can be purchased separately.

- **The blood vessels are consumable items.**

If carrying out repeated injections and a large number of holes are made in the blood vessels, purchase replacement blood vessels (standard size).

8 Troubleshooting

| No. | Symptom or problem | Remedy | Reference page |
|-----|---|---|-----------------------|
| 1 | The product does not operate when the power switch is turned on | Check that the batteries are inserted the correct way.(There may be no batteries inserted or the polarities may be incorrect.) | "6 How to use" (P.31) |
| | | Check that the AC adapter is connected correctly to the tank with integrated pump and to the wall outlet. | "6 How to use" (P.31) |
| 2 | When the power switch is turned on, the red indicator flashes. | Check that the blood vessel connection tubes are not bent and that nothing is sitting on top of the blood vessel connection tubes. (Arm number indicator flashes red) | "6 How to use" (P.39) |
| 3 | When the power switch is turned on, the indicator flashes yellow. | Replace the batteries. | "6 How to use" (P.31) |
| 4 | Backflow does not occur. | Check that the blood vessel connection tubes are connected correctly and that they are securely installed. | "6 How to use" (P.36) |
| | | If two arm models are connected, check that the unit select button is set to "2". (The indicator illuminates blue.) | "6 How to use" (P.38) |
| | | Check that the cap of the tank with integrated pump is securely tightened. | "6 How to use" (P.37) |

| No. | Symptom or problem | Remedy | Reference page |
|-----|--|---|-----------------------|
| 5 | Water leaks out from the cap of the tank with integrated pump. | Check that the cap is securely tightened. | "6 How to use" (P.37) |
| 6 | There is a disagreeable odor. | Stop using the product and contact Sakamoto Model. (The inside of the main unit may be corroded.) | - |

If any problems are not fixed when the remedies given above are carried out, contact Sakamoto Model.

9 Specifications

■ Overall product

| | | |
|---------------------------|------------|-----------------------------|
| Power supply | AC adapter | AC 100 – 240 V 50/60 Hz |
| | Batteries | DC 9 V |
| Power consumption | | 11 W |
| Weight | | 4.3 kg |
| Continuous operating time | | 3 hours or less recommended |

■ Arm model

| | |
|------------|--|
| Dimensions | W 60 cm × D 11 cm × H 12 cm (Dimensions are approximate.) |
| Weight | 840 g |

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URL <http://sakamoto-model.co.jp>

E-Mail trade@sakamoto-model.com

Product specifications and appearance are subject to change for improvement without notice.

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- Some of the components used in this product are manufactured by other companies, and product names, etc. for these components may be trademarks or registered trademarks of the manufacturers.