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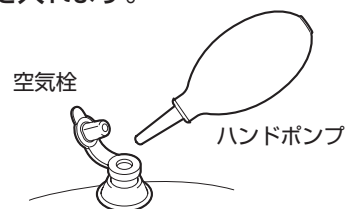
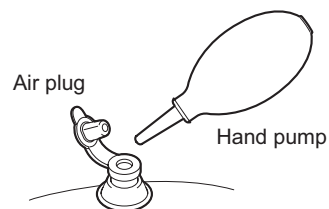
1 Quick Preparation Guide

This guide describes the mechanism and how to start using the "Self Study I.V. kit BASIC".

Be sure to read the "Cautions on use" (P.3) before using this product.

1

Use the hand pump to pump air into the arm model.

Injection pad

Injection area

Arm model

Attach to underside

Double-sided adhesive sheet

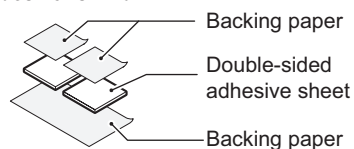
Double-sided adhesive sheet

Tube (short)

(2)Clamp blue

2

Attach double-sided adhesive sheet at two place on the underside of the arm model, and secure the arm model to a desk or similar.

Backing paper

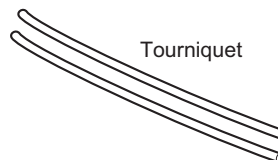
Double-sided adhesive sheet

Backing paper

The double-sided adhesive sheet and the backing paper can be repeatedly reused, so store them safely without throwing them away.

- 1 -

Tourniquet

When wrapping the tourniquet around the arm model, be careful not to wrap it around the tubes as well.

4

Pour water into the tank, and then operate the clamps so that the tubes fill with water.

Clamp operation

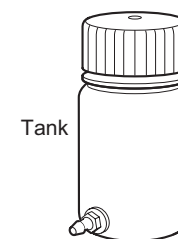
Close clamp (1) white → Pour water into tank → Open clamps (1) white and (2) blue
 Water passes through tube → Close clamp (2) blue

Only use water or artificial blood as the fluid for injecting.

If it tips over, the water will flow out.

Tank

3

Connect the tube (long) and the tube (short) to the injection pad and the tank.

5

Attach the injection pad to the arm model and start training!

- Do not inject outside the injection area.
- Always use fresh needle tips in the syringes when practising.

Putting away

- Drain any water that remains inside the tank and the tubes, and put all of the parts in a shaded place to fully dry.
- Peel off the pieces of double-sided adhesive sheet from the arm model, and then re-attach the backing paper and store the pieces away safely.
 If the adhesive force of the double-sided adhesive sheet becomes too weak, wash it with water to remove any contamination.

- 2 -

2 Cautions on use

In order to use this product safely, be sure to observe the following.

(Items which are particularly important are marked with " ■ ".)

● **Handle these items gently.**

- Do not touch them with sharp objects such as knives, scissors or needles.
(The injection area can be pierced with the injection needle.)
- Do not pull or twist items with a strong force.
- Do not subject items to severe shocks

● **Always use fresh needle tips in the syringes when practising.**

If you use a syringe with a needle that has a bent or damaged tip, it may cause the blood vessels to become damaged or the injection pad to become degraded.

● **Only use water or artificial blood as the fluid for injecting.**

If a fluid other than those mentioned above is used, damage or disagreeable odors may result.

Use either tap water or distilled water for injection use.

To create artificial blood, add 1 to 2 drops of the artificial blood concentrate to water.

● **Do not process or modify the product.**

Do not make holes in the product, or process or modify it so that other items can be installed to it, etc. Damage or unexpected problems may occur.

● **There are limits to the effectiveness of simulator training.**

This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee that training using this product will provide the same level of expertise as actual treatment.

● **Do not use this product for other applications.**

Do not use this product for applications other than the original application for which it is designed. The product may become damaged or injury may result.

● **This product is a consumable item.**

Even if it is used correctly according to this instruction manual, it may deteriorate and become damaged over time.

3 Cleaning and storage

Be sure to observe the following when cleaning and storing the product.

● **Clean the arm model and the surface of the injection pad by wiping them gently with a soft cloth.**

- If they are extremely dirty, wipe them with a cloth which has been moistened with diluted detergent or alcohol, and then wipe them again with a dry cloth.
- Do not use benzine, thinner or other solvents.
Otherwise, it may damage the product or cause the surface to deteriorate.
- If you write on the arm model or injection pad using a ballpoint pen or similar object, the surfaces will become smeared with ink which cannot be removed.

● **Dry out the inside of the product before storing it.**

You should let the main unit stand for about half a day to a day to allow the insides of the parts to fully dry out. If the product is stored away without fully drying it first, disagreeable odors will result.

● **When not using the product, release the air from the arm model.**

Leaving the air inside the arm model when not in use may cause the product to deteriorate quickly, and air leaks may result.

● **Place the product into its special storage bag.**

Use the special storage bag which the product was supplied in at the time of purchase.

● **Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.**

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, discoloration or deterioration may occur.

SAKAMOTO MODEL C O R P O R A T I O N

34, Higashitakagicho, Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN

Tel +81-75-701-1135

Fax +81-75-722-5638

URL <http://sakamoto-model.co.jp>

E-Mail info@sakamoto-model.com

- Product specifications and appearance are subject to change for improvement without notice.
- Some of the components used in this product are manufactured by other companies, and product names, etc. for these components may be trademarks or registered trademarks of the manufacturers.